## Harvard Polycarboxylat Cement

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#### Zink-Polycarboxylat-Zement

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#### Zinc polycarboxylate cement

##### Composition:

1 g powder contains:  
 890 mg zinc oxide, 89 mg magnesium  
 oxide  
 1 g liquid contains:  
 400 mg polyacrylic acid

##### Areas of Application:

Fixation of crowns and inlays, cavity linings

##### Dispensing Instructions:

Fixation consistency:  
 2,9 g powder : 1,0 g liquid  
 Cavity lining consistency:  
 3,6 g powder : 1,0 g liquid  
 Dispensing and testing climate for testing in  
 accordance with DIN EN ISO 9917:  
 Mix quantities indicated at 23 ± 1°C and a  
 relative humidity of 50 ± 10%.

##### Processing Recommendations:

A clean, dry glass plate at approx. 20 °C  
 (68 °F) and a non-discolouring spatula are  
 recommended for mixing.

##### Mixing:

Divide the measured quantity of powder  
 into 3 portions (1/2 1/4 1/4). Beginning with  
 the largest quantity, stroke the powder into  
 the measured liquid within 60 seconds. The  
 mixture should appear thicker as usual for  
 zinc phosphate cement, yet it is just as  
 easy to work. The fixation consistency has  
 been reached when the peak formed on  
 lifting the spatula slowly falls back into the  
 paste. The filling consistency has been  
 reached when on lifting the spatula, the  
 peak drawn out forms a small hook and  
 does not sink back into the paste. Discard  
 remaining powder or impure liquids.

**Mixing Time:** 60 seconds

##### Processing Time from start of mixing

(at 23 °C, 72 °F):

For fixation: 2 minutes  
 For cavity linings: 1 minute 30 seconds

##### Setting Time from end of mixing:

2 30 – 6 00 minutes

##### Special Informations:

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp
- Apply the thinnest possible layer for fixations
- The cement must be applied to the dry tooth surface and must set without contact to moisture
- Remove excess material from neighbouring teeth, facets etc. immediately
- Do not expose liquid to sunlight and frost
- Close containers immediately after removing material
- Clean instruments and glass plate with water before the cement sets

- Zinc polycarboxylate cement can be dissolved using a cement-solvent after setting

##### Storage information

Store at 2 – 28 °C (36 – 82 °F). Do not use after expiry date.

##### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

##### Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

##### Only for dental use!

##### Store product out of reach of children!

Harvard Polycarboxylat Cement meets the requirements of DIN EN ISO 9917. The instructions for use has to be kept for the duration of the application.

Order-No.	Article
	Powder
	Single powder, 35 g
7031503	3 yellowish-white
7031504	4 light yellow
	Liquid
	Clinic powder, 100 g
7031203	3 yellowish-white
7031204	4 light yellow
	Liquid
7031600	Single liquid, 15 ml
7031300	Clinic liquid, 40 ml

**Harvard Dental International GmbH**  
 Margaretenstr. 2 – 4  
 15366 Hoppegarten, Germany  
 Tel: +49 (0) 30/99 28 978-0  
 Fax: +49 (0) 30/99 28 978-19  
 info@harvard-dental-international.de  
 www.harvard-dental-de

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