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**Instructions for Use EN**

**Flowable composite for blocking and masking**

Harvard Block'n Mask Flow is a flowable light cure radiopaque composite that is used in combination with Harvard MultiChrome or Harvard MultiChrome Flow to mask discolored teeth. Harvard Block'n Mask Flow contains dimethacrylate based resins and inorganic fillers < 0.2 µm. The total filler content is 67 % by weight and 57 % by volume. Radiopacity of Harvard Block'n Mask Flow is 150 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).

**Indications Harvard Block'n Mask Flow**

- Masking of the oral cavity in case of class III and IV cavities
- Masking of tooth discoloration

**Contraindications**

The placement of Harvard Block'n Mask Flow is contraindicated if a dry working area or the recommended application technique are not possible. Also do not use the composites if the patient is known to be allergic to any of the ingredients.

**Side effects**

In singular cases, Harvard Block'n Mask Flow may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

**Incompatibility with other materials**

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

**Application**

**1. Preparation of tooth**

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation.

**2. Cavity preparation**

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

**3. Application of a bonding agent**

Apply a common adhesive for enamel and dentine bonding according to the corresponding manufacturer instructions. The application of the adhesive InterLock® ONE is recommended. Follow the instructions for InterLock® ONE.

**4. Application and curing of Harvard Block'n Mask Flow**

In case of cavities of dark or discolored teeth use Harvard Block'n Mask Flow to ensure that the restoration may not appear dark. Apply Harvard Block'n Mask Flow by using suitable instruments. Matrix strips may be used.

**Preparation of the syringe:**

Screw off the cap of the syringe and substitute it by one of the supplied needle tips (by turning clockwise). **Store the screw cap!** After use discard the needle tip and attach the cap to the syringe again. For hygienic reasons needle tips are for single use only.

Apply Harvard Block'n Mask in **layers of max. 1 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 nm) with a **light intensity of at least 1000mW/cm<sup>2</sup>**, cure each increment separately for **20 seconds**.

Hold the light emission window as close as possible to the filling material.

Finish the restoration with Harvard MultiChrome or Harvard MultiChrome Flow according to the instructions.

**5. Additional informations/warnings**

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

**Storage information**

Storage temperature 4–25°C (39–77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

**Warranty**

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

**Limitation of liability**

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**For dental use only!**

**Store product out of reach of children!** Harvard Block'n Mask Flow conforms to the DIN EN ISO 4049, type 1, class 2, group 1. Keep the instructions for use for the duration of the application.

**Order-No.** 7100006  
**Article** Harvard Block'n Mask Flow, 2 x 1 ml syringe, 8 needle tips

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