




# Harvard ZirconCore

## Dual cure core build-up and post cementation composite


- Excellent consistencies for both indications:  
**core build-up** and **post cementation**
  - “Cuts like dentin“
  - Very good mechanical properties for permanent restorations
  - Contains a substantial amount of Nano-zirconia
  - Fluoride release
  - Thixotropic properties
  - Very good radiopacity
  - Self-mixing material with the Minimix syringe
- 
- For optimal results, use with  
**Harvard InterLock® ONE /**  
**Harvard InterLock® ONE Self Cure Activator**  
or with **Harvard Bond SE Dual**

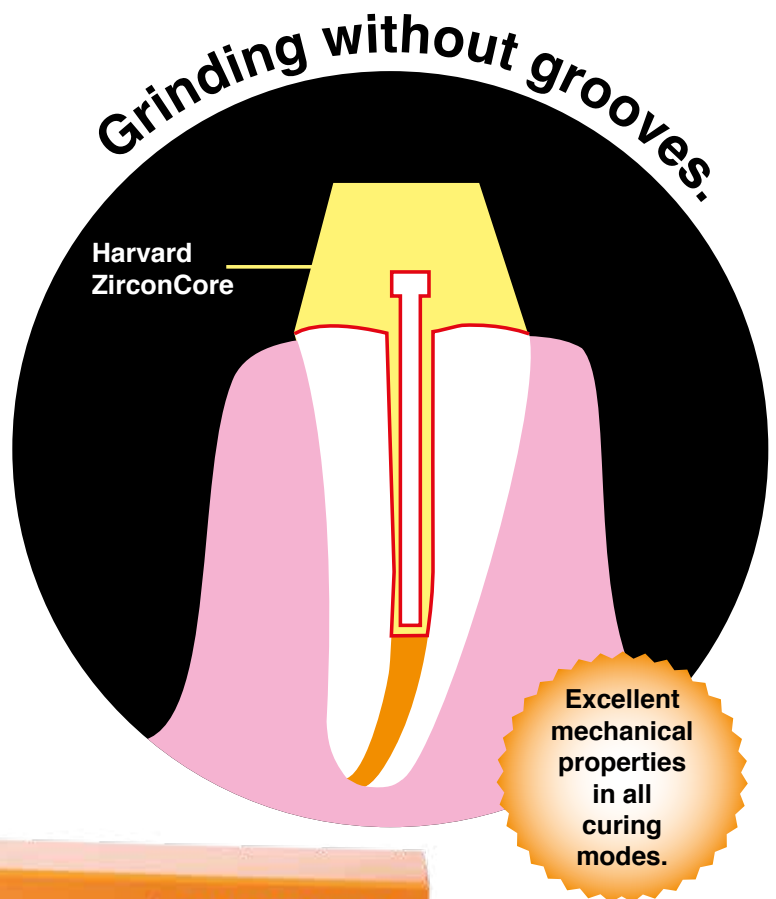

## Physical Properties

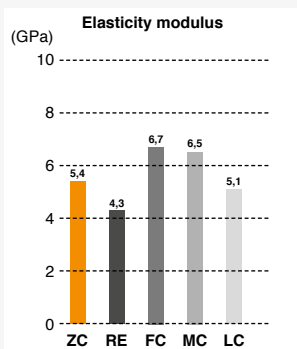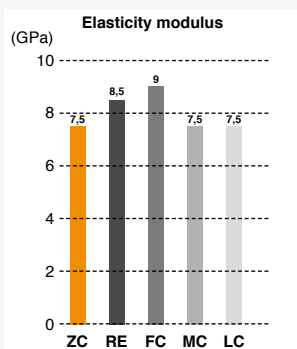

**ZC** Harvard ZirconCore  
**RE** Voco Rebuilda  
**FC** Dentsply FluoroCore 2+  
**MC** Ivoclar Vivadent MultiCore Flow  
**LC** DMG LuxaCore Z Dual Automix

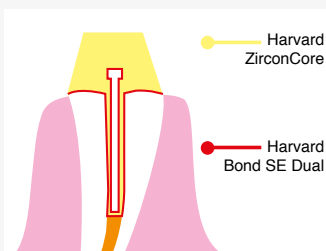

## Post cementation and core build-up



1. Prepared and bonded root canal
2. Application into the root canal
3. Placement of the post
4. Light cured core build-up
5. Shaped core build-up

### Technical data

Working time (23 °C / 73 °F)	1:30 min
Setting time (self cure)	3:30 min
Light cure (optional)	40 sec

### Article information

Article information	Order no.
<b>Harvard ZirconCore</b> 5 ml minimix syringe, 10 mixing tips, 10 intra tips long, Shade A3	7083600
<b>Harvard Mini 1:1 O-Brown</b> refill bag with 50 mixing tips	7083610
<b>Harvard IntraTips long</b> refill bag with 50 intra tips long	7083620
<b>Harvard Bond SE Dual</b> 2 x 5 ml bottle, 50 Microbrush®, 5 mixing wells	7083602
<b>Harvard Bond SE Dual Refill</b> 2 x 5 ml bottle	7083605

Liability is excluded for all printing errors and omissions. Before using our Harvard products, the respective directions for use should be noticed in every case. All measurements are internal measurements of Harvard Dental International.

Harvard Distribution Partner.

We recommend **Harvard Bond SE Dual**: the dual cure, self etch adhesive

1892  
seit  
HARVARD



# HARVARD®

Marke und Qualität seit 1892

Harvard Dental International GmbH  
Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany  
Phone: + 49 (0) 30/99 28 978-0  
Fax: + 49 (0) 30/99 28 978-19  
info@harvard-dental-international.de