Harvard Bond SE Dual

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Harvard Bond SE Dual

**Self-etching, dual-curing adhesive
(two component system)**

The dual-curing adhesive **Harvard Bond SE Dual** was developed for direct restorations and core build-ups with dual or self-curing composites, e.g. **Harvard ZirconCore**. It is equally suited for light-curable composites, compomers, and resin modified glass ionomer cements.

The easily applied dual-curing adhesive guarantees a durable and secure adhesion between dentine and/or enamel and restoration material, even on slightly moist tooth surfaces.

Harvard Bond SE Dual consists of the components Part A and Part B.

Indications

Bonding of light, self and dual-curing composites, compomers, and resin modified glass ionomer cements.

Contraindications / side-effects

In rare cases of patients with a hypersensitivity to one of the components, **Harvard Bond SE Dual** can lead to sensitisation. In these cases the use of the material should be discontinued.

Irritation is possible in direct contact with the pulp. Therefore the bottom surface of the cavity must always be covered with a thin layer of calcium hydroxide cavity liner to protect the pulp.

Interaction with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

1. Preparation

The use of a rubber dam is recommended.

2. Preparation of the cavity

Before beginning the preparation, the tooth must be cleaned of residue. Prepare the cavity in such a way that the least amount of healthy tooth material is lost. Slightly taper the cavity margin on the tooth enamel (0.5-1.0 mm) in order to enlarge the bond area in the tooth enamel and thereby increase the bonding stability.

3. Protection of the pulp

For deep drilling in the cavity areas close to the pulp, cover with a thin layer of calcium hydroxide cavity liner.

4. Etching, priming, bonding

Phosphoric acid etching of the enamel and dentine is not necessary with **Harvard Bond SE Dual**.

One drop of Harvard Bond SE Dual PART A and one drop of Harvard Bond SE Dual PART B were combined in a mixing pallet and mixed for 5-10 seconds.

Apply the homogeneous mixture generously with the brush onto the slightly wet enamel- and dentine surfaces (as they are usual in clinical procedures) for 30 seconds with agitation. The material should build a homogeneous layer. Air thin gently for 10 seconds to remove the volatile components and to disperse the adhesive. Then light cure for 20 seconds with a dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) and place the restorative material.

For a maximal adhesion it is strongly recommended to use the light cure mode.

If light cure is absolutely impossible, the adhesive will also cure in the autocure mode. After application of the adhesive air thin gently **to remove all volatile components**. There must remain a sticky layer. Then apply the restorative material directly.

Notes:

Do not interchange lids of the bottles, because this can lead to a cross-contamination of the liquids.

Core build-ups

Harvard ZirconCore can be directly applied in accordance with Manufacturer's directions.

Fillings

For an optimal result, a flowable composite in a thin layer should first be applied and light-cured. Subsequently a mouldable composite can be applied in accordance with the instructions of the manufacturer.

Warnings

- Uncured material can cause irritation and lead to hypersensitivity to methacrylate.
- Avoid contact with skin, mucous membranes and eyes.
- In case of contact with skin, immediately wash with water and soap. In case of contact with the eyes, immediately rinse with plenty of water and as appropriate consult a physician!
- Standard medical gloves offer no protection against the sensitizing effect of methacrylate.

Storage

Storage temperature 4-25 °C (39-77 °F)! Avoid storage in direct sunlight. Do not use after expiration date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental Internationals sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Only for dental use!

Store product out of reach of children!

The instructions for use has to be kept for the duration of the application.

Order-No.	Article
7083602	Harvard Bond SE Dual 2 x 5 ml bottle, 50 Microbrush®, 5 mixing wells
7083605	Harvard Bond SE Dual Refill, 2x5 ml bottle