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## Harvard CalciumHydroxide

### Calcium hydroxide paste based on water

Harvard CalciumHydroxide is a radiopaque water-based calcium hydroxide liner.

#### Indications

- Direct pulp capping when the pulp is opened or a pulpotomy performed.
- Indirect pulp capping for the treatment of caries profunda.
- Temporary filling of the root canal.
- Pulp protection against acids when using cements (e.g. Harvard Cement).

#### Contraindications

- Direct pulp capping after exposure of the pulp in carious dentin if:
  - the area of exposure is larger than 1 mm<sup>2</sup>
  - a completely dry field cannot be established
  - the treatment of the tooth cannot be completed in one appointment
  - the tooth concerned is already causing discomfort
- direct and indirect pulp capping if the pulp is damaged irreversible
- the material should not be used in patient with a known allergy to any of the ingredients

#### Application

Clean and dry the cavity.

If desired, the application needle might be bend in a slight arc to facilitate the application.

#### Handling precaution

Before each application squeeze out the first small drop on a mixing pad to ensure proper and controlled handling.

Apply Harvard CalciumHydroxide in thin layers. Allow to dry briefly or dry carefully with an oil and water free stream of air. Do not apply Harvard CalciumHydroxide to the cavity edges and remove any paste sticking to them. Then proceed with the restoration.

For temporary root canal fillings, first prepare and clean the root canal (e.g. hydrogen peroxide 3% or sodium hypochlorite solution) and insert Harvard CalciumHydroxide with a suitable instrument. Avoid overfilling (apical) and trapping of air.

#### Important notes

Stick the application needle of the syringe **immediately after use** through the septum into the anti-dry protector. The chemical and physical properties of the anti-dry protector prevent hardening of Harvard CalciumHydroxide in the application tip and the anti-dry protector acts also as disinfecting agent. For the **next** application take a **new application needle**.

**Do not use hardened paste.**

#### Side-effects

If the root canal is overfilled, discomfort may result (pressure sensitivity and/or swelling) but, as a rule, this will subside within 24-48 hours.

Contra-indications or interactions are not known.

#### Warnings

- When using Harvard CalciumHydroxide wear protective glasses and cover the patients eyes.
- Harvard CalciumHydroxide is irritant. In case of contact with eyes a risk of serious damage to the eyes exists.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- If accidental skin contact occurs, wash immediately with water.

#### Storage

Do not store at less than +8° C!

Do not use after expiry date.

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

#### Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

#### For dental use only!

#### Store product out of reach of children!

Keep the instructions for use for the duration of the application.

#### Order-No. Article

7071261	Harvard CalciumHydroxide 2 ml syringe, 3 needle tips
7071262	Harvard CalciumHydroxide 20 SingleBlis ea. 0,3 g