## Harvard CalciumHydroxide LC

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### Instructions for Use EN

#### Calcium hydroxide paste, light cure

Harvard CalciumHydroxide LC is a light cure, radiopaque liner with calcium hydroxide.

#### Indications

- indirect pulp capping
- lining of cavities for following filling procedures

#### Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the liner. Neither store the liner in proximity of eugenol containing products, nor let the liner allow coming into contact with materials containing eugenol.

#### Application

Clean and dry the cavity thoroughly. Avoid contamination. Rubber dam is the recommended method of isolation.

#### Note:

In direct proximity to the pulp, separated by a thin layer of dentine only, application of an aqueous calcium hydroxide liner (e.g. Harvard CalciumHydroxide) or Harvard MTA is recommended.

To enhance bonding of Harvard CalciumHydroxide LC use a bonding agent. Etching with Harvard Etch and application of Harvard Bond TE Mono according to the corresponding user instructions is recommended. In areas with a thin dentin layer over the pulp do without bonding (pulp protection).

Then Harvard CalciumHydroxide LC can be applied directly from the syringe.

Screw off the cap of the syringe and substitute it by a NeedleTip.

Apply Harvard CalciumHydroxide LC in a thin layer and light cure for 40 sec. In case of layers more than 1 mm cure each layer separately for 40 sec.

After curing apply the filling material according to manufacturer instructions.

#### Warnings

- When using Harvard CalciumHydroxide LC wear protective glasses and cover the patient eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin and mucous membrane. If accidental contact occurs, wash immediately with water.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Do not use any resin to adjust viscosity of composite restorative material.

#### Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Close syringe immediately after use to avoid exposure to light.

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

#### Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

#### Only for dental use!

#### Store product out of reach of children!

The instructions for use has to be kept for the duration of the application.

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