Harvard Ionoglas Fill Extra OptiCaps®

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Harvard Ionoglas Fill Extra OptiCaps®

Esthetic self cure glass ionomer cement for restorations

Harvard Ionoglas Fill Extra OptiCaps® is an esthetic restorative glass-ionomer cement in OptiCaps® with improved mechanical properties. Besides its high fluoride release and high biocompatibility as glass ionomer cement Harvard Ionoglas Fill Extra also has good chemical bonding to dentin and enamel and a tight seal at the dental margins. Therefore it requires no enamel and dentin etching. Harvard Ionoglas Fill Extra is radiopaque.

Harvard Ionoglas Fill Extra OptiCaps® can quickly and simply be activated and extruded with the Harvard Applier OptiCaps®. Capsule mixing (mixing time 10 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min. Application can be done directly from the capsule.

Indications

- Deciduous teeth: final restorative for Class I, II and V (according Black)
- Long term restorative in non-load bearing areas of Class I and II
- Class V restorations
- Intermediate restorations
- Sandwich material for heavy stress bearing Class I and II cavities

Contraindications / side-effects

Do not use Harvard Ionoglas Fill Extra with patients who show an allergy to the material. If allergic reactions occur, further treatment with Harvard Ionoglas Fill Extra must be discontinued and the patient must be advised to consult a physician.

Do not allow the liquid or cement mixture to contact the oral tissues or skin. In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water.

Avoid eye contact of the liquid or cement mixture. In case of contact, immediately flush with water and seek medical advice.

Interaction with other materials

Do not mix the powder or liquid of Harvard Ionoglas Fill Extra with any other glass-ionomer product.

Application

1. Preparation

Prepare the tooth to be treated in the accustomed manner.

Clean the cavity preparation with pumice and water. Rinse thoroughly and dry, but **do not** desiccate.

If desired, place a matrix band.

By using a cotton pellet apply a dentin conditioner (e.g. Harvard Ionoconditioner) for **20 seconds** to remove the smear layer. Rinse the dentin conditioner with water and dry it in an airstream, but **do not** desiccate.

Pulp capping with Harvard Ionoglas Fill Extra is **contraindicated**. To deep areas close to the pulp apply a small amount of a calcium hydroxide liner (e.g. Harvard CalciumHydroxide).

2. Mixing of Harvard Ionoglas Fill Extra OptiCaps®

The activation and mixing of the OptiCaps® is done in correspondence with the directions of the "Instructions for the activation and mixing of Harvard Ionoglas Fill Extra OptiCaps®".

Mixing time for Harvard Ionoglas Fill Extra OptiCaps®: 10 seconds

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent the application of the material.

All the cement required should be dispensed from the OptiCaps® within **15 seconds** after end of mix.

Harvard Ionoglas Fill Extra sticks to metal instruments and should be washed off with cold water prior to setting.

3. Filling

Remove moisture from the cavity surface either with a cotton pellet or very gently with an air syringe blast. It is essential not to desiccate the dentin and enamel surfaces.

Apply the mixed Harvard Ionoglas Fill Extra directly out of the capsule within the working time into the prepared cavity.

Please see to it, that no air bubbles will be incorporated.

Form the material with a placement or forming instrument. Optional use a matrix strip to form the surface.

Working time from start of mix at 23°C (74°F): **1:30 min**

Net setting time at 37°C (99°F): **3:30 min**

Notes

Slightly wet instruments simplify filling and modelling.

Higher temperatures will shorten the working time, lower temperatures will prolong the working time.

An overextended working time reduces adhesion to enamel and dentin.

If a matrix strip is placed, don't remove it before end of net setting time.

After setting, immediately apply **Harvard Ionocoat LC** and light cure for **20 seconds**.

Close bottle immediately after use!

4. Finishing

Final finishing and polishing can begin from about **6 minutes** after start of mixing. Smooth with fine diamond burs, then polish with finishing and polishing discs with graded grain size.

Spray preparation dust away with water and dry the surface in an oilfree airstream.

Apply a thin layer of **Harvard Ionocoat LC** to the final finished surface of the restoration. Immediately light cure for **20 seconds**.

Close bottle immediately after use!

Instruct the patient not to expose the restoration to any pressure for one hour.

Storage information

Store Harvard Ionoglas Fill Extra in the original aluminium foil in a cool place at 4-25 °C (39-77 °F). Do not use after expiry date.

OptiCaps® are for single use only

Average net content per OptiCaps®: 0.5g

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!

Harvard Ionoglas Fill Extra conforms to the DIN EN ISO 9917-1.

Keep the instructions for use for the duration of the application.

Order-No. Article


50 OptiCaps® ea. 0,5 g

7052252 Shade A2


7052253 Shade A3

7052254 Shade A3,5


7092000 Harvard Applier OptiCaps®

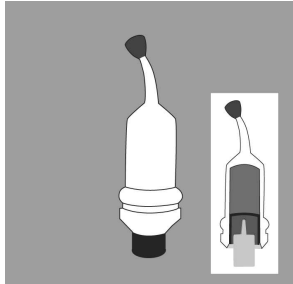Instructions for the activation and mixing of Harvard OptiCaps®

Harvard OptiCaps® before activation.

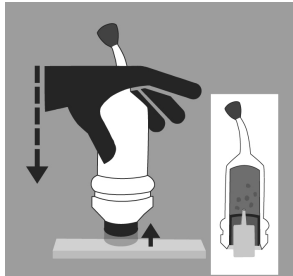
For activation of the Harvard OptiCaps®, press the plunger on a hard and plane surface to the end into the capsule.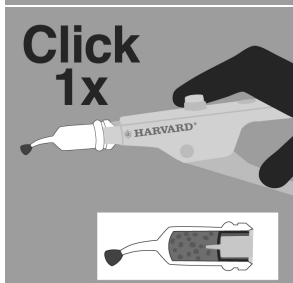


Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps® and click once.


Note: The plunger must be at the same level as the bottom of the capsule.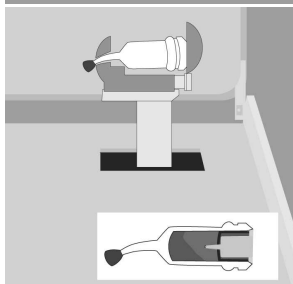


Insert the Harvard OptiCaps® into a mixer, close lid and mix immediately for **10 seconds** at 4,300 oscillations/minute.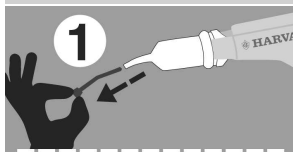

Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps®. Remove the pin from the nozzle. If you leave it, the capsule can burst.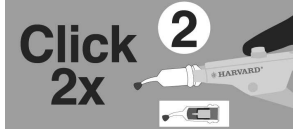

Pull the lever 2 times (2 clicks) to prime the the Harvard OptiCaps®.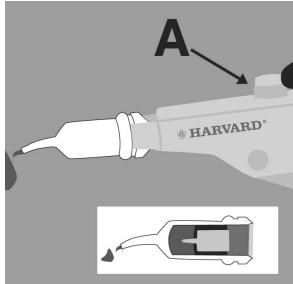

Extrude the mixed material on a glass plate or apply directly.

Unlock the Harvard Applier OptiCaps® (press button A) and remove the Harvard OptiCaps®.

The optimal amount of mixed material is guaranteed only with the Harvard Applier OptiCaps® (Order-No. 7092000).

For the selection of a suitable capsule mixer, our sales and marketing colleagues are gladly available to you.