Harvard Bond SE Mono

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Instructions for Use EN

Light-curing self-etching one component adhesive

Harvard Bond SE Mono is a self-etching light cure one-component adhesive.

The simple to use material is applied on moist tooth surfaces and guarantees a long lasting and safe adhesion between dentin /enamel and the light cure restorative.

Harvard Bond SE Mono is compatible with all visible light cure methacrylate based composite restorative materials.

Indications

Harvard Bond SE Mono is designed for bonding of light cure composites to enamel and dentin

Contraindications

The placement of Harvard Bond SE Mono is contraindicated if a dry working area or the recommended application technique are not possible. Also do not use Harvard Bond SE Mono if the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, Harvard Bond SE Mono may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection the cavity floor in cases of deep excavations should be covered with a thin layer of calcium hydroxide material.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the adhesive. Neither store the material in proximity of eugenol containing products, nor allow not polymerized adhesive to come into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material (e.g. Harvard CalciumHydroxide).

4. Application of Harvard Bond SE Mono

Harvard Bond SE Mono SingleDose: Pierce the supplied Microbrush® vertical through the aluminium foil.

Apply Harvard Bond SE Mono with a brush onto the **moist enamel and dentin surfaces for 30 seconds** with agitation.

The material should build a homogeneous layer. Air thin gently for **10 seconds** to remove the volatile components and to disperse the adhesive. Take care, that the adhesive is not removed from the tooth surface by a strong air stream.

Do not touch the treated surface!

Then light cure with a suitable polymerization unit (wavelength range 400-500 nm, light intensity of at least 500mW/cm²) for **20 seconds** before placement of a light cure composite.

5. Application of a composite

Refer to manufacturer instructions for placement of the light cure restorative material.

Note for direct restorations:

Best results are obtained with application of a thin layer of a light cure flowable composite (e.g. Harvard PremiumFlow) followed by the application of a moldable composite (e.g. Harvard PremiumFill) according to the corresponding user instructions.

6. Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Harvard Bond SE Mono SingleDose is for single use only

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!
Keep the instructions for use for the duration of the application.

Order-No.	Article
7083601	Harvard Bond SE Mono, 5 ml bottle, 25 Microbrush®, 5 mixing wells
7083604	5 ml bottle
7083609	50 SingleDose ea. 0.15 ml, 50 Microbrush®

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Microbrush® is not a registered trade mark of Harvard Dental International GmbH.