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Harvard PremiumFlow[®] is a light cure nano-filled flowable composite for restorations with high demands for esthetics and durability. Beside its low shrinkage Harvard PremiumFlow[®] has a controlled flowable consistency for a very comfortable application.

Harvard PremiumFlow[®] is based on urethanedimethacrylate resin and inorganic filler particles <1.0 µm. The filler content: is 62 % by weight and 48 % by volume.

Indications

- Class V restorations (cervical caries)
- Anterior class III and IV restorations
- All kind of small restorations
- Lining of cavities
- Repair of composite or ceramic facings

Contraindications

The placement of Harvard PremiumFlow[®] restorations is contraindicated if a dry working area or the recommended application technique are not possible. Also do not use Harvard PremiumFlow[®] if the patient is known to be allergic to any of the ingredients.

Side effects

In singular cases, Harvard PremiumFlow[®] may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite come into contact with materials containing eugenol.

Application

1. Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching. Ascertain the tooth shade while teeth are still moist and select the appropriate Harvard PremiumFlow[®] shade.

Harvard PremiumFlow[®] is shaded according to VITA[®]-Shades.

2. Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of calciumhydroxide material (e.g. Harvard CalciumHydroxide).

3. Application of a bonding agent

Apply a common adhesive for enamel and dentine bonding according to the corresponding manufacturer instructions.

The application of the adhesive Harvard Interlock is recommended. Follow the instructions for Harvard InterLock.

4. Application and curing of Harvard PremiumFlow[®]

Apply Harvard PremiumFlow[®] in the selected shade. Place it by using suitable instruments. Matrix strips may be used.

For an optimum result apply Harvard PremiumFlow[®] in **layers of max. 2 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400 – 500 nm) with a light intensity of at least 1000mW/cm², cure each increment as follows: Lighter shades (e.g. A1, A2, C2) **20 seconds**

Darker shades (e.g. A3.5, B3) **30 seconds**
Hold the light emission window as close as possible to the filling material.

5. Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss with polishing discs or polishing brushes.

6. Additional informations

- If Harvard PremiumFlow[®] in OptiTips[®] is applied directly in the mouth, for hygienic reasons the OptiTip[®] is for single use only.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized Harvard PremiumFlow[®] may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA[®] is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!
Harvard PremiumFlow[®] conforms to the ISO 4049.

The instructions for use has to be kept for the duration of the application.

Order-No.	Article
Harvard PremiumFlow [®] Refill	2 x 1 ml syringe, 8 needle tips
7082600	Shade A1
7082601	Shade A2
7082602	Shade A3
7082603	Shade A3.5
7082607	Shade B2

20 OptiTips [®] ea.	0,25 g
7082700	Shade A1
7082701	Shade A2
7082702	Shade A3
7082703	Shade A3.5
7082704	Shade B2

Harvard PremiumFlow[®]

Harvard Dental International GmbH
Margaretenstr. 2 – 4
15366 Hoppegarten, Germany
Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de