



## Mixing Advices for Harvard Cement



1

**Tip:**  
use a cold glass plate,  
especially in warm  
countries.

Dispense onto a clean, dry glass plate powder and liquid at approx. 23 °C (73 °F).



2

Divide into 4 portions as follows: 1/2, 1/4, 1/8, 1/8.



3

Mixing: start first 1/8 with the whole liquid quartely within 15 seconds.



4

Add second 1/8 and mix for 15 seconds while spreading.



5

Draw 1/4 into the mixture.



6

Mix while pressing with flat spatula in the next 30 seconds.



7

Mix with the remaining half portion for 30 seconds to obtain an homogen mass.



8

Use the entire surface of glass plate.



9

Ready-for-use cement mix within 90 seconds.

### Harvard Cement normal setting:

For luting consistency: powder 1.5 g, liquid 1.0 g  
For cavity lining consistency: powder 2.1 g, liquid 1.0 g

### Harvard Cement quick setting:

For luting consistency: powder 1.8 g, liquid 1.0 g



Please observe the instructions for use!



Made in Germany