



# Harvard TEMP Cem Esthetic

Translucent eugenol-free zinc oxide cement  
for temporary luting.

The aesthetic cement with a high transparency.

- Easy removal of crowns and bridges
- Easy removal of excess material
- After removal no residual cement remains on the core
- Low film thickness for a perfect fit
- Eugenol-free
- Comfortable handling and time saving with the minimix syringe

Translucency for high aesthetics!




Translucent!



# Aesthetic and comfortable.

## Aesthetic characteristics

Harvard TEMP Cem Esthetic achieves a very high translucency and natural aesthetics.

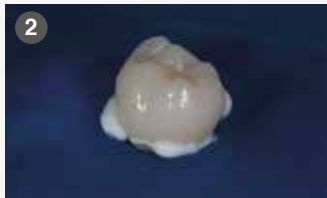

1. Application of material into the crown
2. Temporary crown cementation
3. **Easy removal** of excess material
4. Fixed crown with Harvard TEMP Cem Esthetic



Harvard TEMP Cem Esthetic



Harvard TEMP Cem



TempBond NE (Kerr)

## Overview translucency of different temporary cements:



Harvard TEMP Cem Esthetic

Harvard TEMP Cem

TempBond NE (Kerr)

## Cement remains in the crown

Harvard TEMP Cem Esthetic allows an easy removal of crowns and bridges. After removal no residual cement remains on the core.



Harvard TEMP Cem Esthetic



Harvard TEMP Cem

## Cement remains in the crown and on the core



TempBond NE (Kerr)

### Technical data

Film thickness	11 µm
Working time ( 23°C / 73°F)	> 1:30 Min.
Setting time	< 3:30 Min.

### Article information

**Harvard TEMP Cem Esthetic**  
5 ml minimix syringe, 10 mixing tips

**Harvard Mini 1:1 S-Brown**  
refill bag with 50 mixing tips

### Order no.

7081104

7091050



# HARVARD®

Marke und Qualität seit 1892

Harvard Dental International GmbH  
Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany  
Phone: + 49 (0) 30/99 28 978-0  
Fax: + 49 (0) 30/99 28 978-19  
info@harvard-dental-international.de

Liability is excluded for all printing errors and omissions. Before using our Harvard products, the respective directions for use should be noticed in every case. All measurements are internal measurements of Harvard Dental International.

Harvard Distribution Partner.