




Harvard TransMatrix

Transparent matrix and bite registration material based on A-silicones

Harvard TransMatrix allows a transparent reprint of the tooth surface and adjacent teeth. This reprint can be used as a customized transparent matrix for a subsequent restoration (e.g. Veneers) with light-curing composites.

- Highly transparent to allow perfect light-curing of composite through the matrix
- Appropriate final hardness and good detail reproduction
- Original shape remains unaltered
- Dimensionally stable over time and after disinfection
- Perfect consistency and thixotropy
- **Fast setting time** (1:20 min in mouth)
- Comfortable for the patient with neutral taste and flavor
- **Time-saving** – directly applied in the mouth
- As well usable as transparent bite registration material

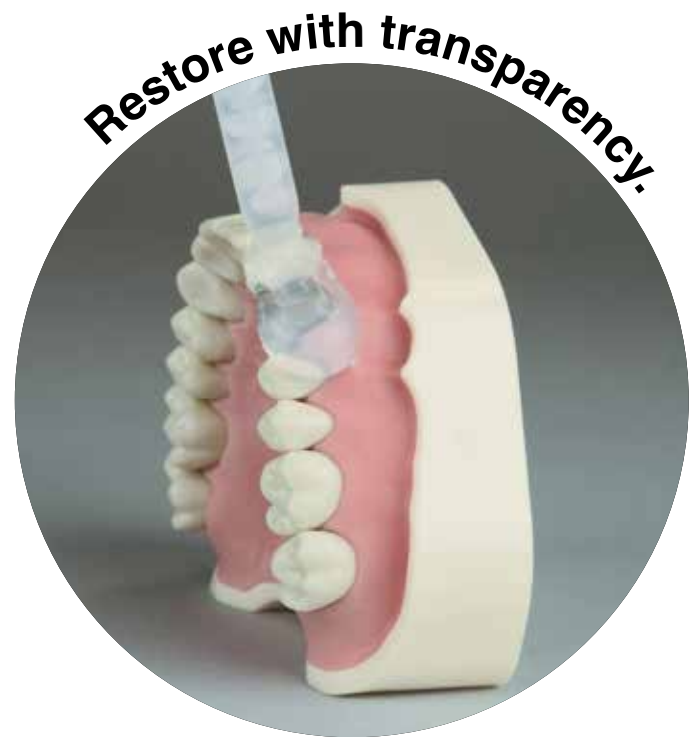

In 12 steps to success.



1 Initial situation



2 Harvard TransMatrix application on the prepared surface and adjacent teeth



3 Fast setting time (1:20 min. intraoral)



4 Complete matrix



5 Preparation with bevelled enamel



6 Etching of the prepared surface with **Harvard Etch**



7 Application of an adhesive system, e.g. **Harvard InterLock ONE**



8 Application of a slightly heated packable composite in the matrix, e.g. **Harvard UltraFill**



9 Correct repositioning of the filled matrix in the oral cavity



10 40 sec light curing through the matrix. Repeat the process after the removal of the matrix



11 Excess removal and polishing with appropriate rotating instruments and if possible finishing strips



12 Finished direct composite restoration

Technical Data

Shore A hardness (after 24 hours) 64

Article information

Order no.

Harvard TransMatrix
2 x 50 ml Automix cartridge,
in transparent shade, 12 mixing tips

7083798

Harvard Auto 1:1 O-Green
Refill bag with 50 mixing tips

7096000

Harvard Dispenser Automix 1:1

7095100

Liability is excluded for all printing errors and omissions. Before using our Harvard products, the respective directions for use should be noticed in every case. All measurements are internal measurements of Harvard Dental International.

Harvard Distribution Partner.



HARVARD®

Marke und Qualität seit 1892

Harvard Dental International GmbH
Margaretenstr. 2 - 4, 15366 Hoppegarten, Germany
Phone: + 49 (0) 30/99 28 978-0
Fax: + 49 (0) 30/99 28 978-19
info@harvard-dental-international.de