Harvard Applier OptiTips®

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DE

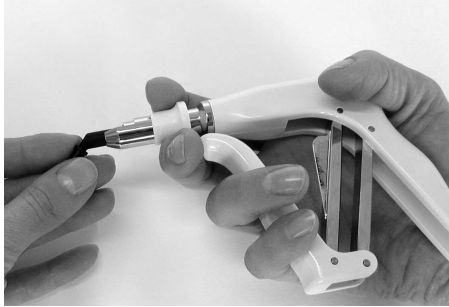
Instructions for use

EN

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The ergonomical designed Harvard Applier OptiTips® is excellent suitable for the easy and reliable application of composite restorative materials in OptiTips® (e.g. Harvard PremiumFill, Harvard PremiumFlow).

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For inserting or changing of OptiTips® pull back the white plastic ring. Now the OptiTip® can inserted easily. Slowly release the ring until the OptiTip® is securely locked into place.

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Now the composite material can be easily and reliably applied. Turn the tip of the OptiTip® to the desired direction and remove the cap.

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The material should easily extrude from the OptiTip®. Move the lever of the Harvard Applier OptiTips® with an even motion. Do not use excessive force! Excessive force may result in uncontrolled flow of material or cause the OptiTips® to eject. If necessary take a new OptiTip®. After application pull back the plastic ring and remove the OptiTip®.

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After use clean and disinfect the Harvard Applier OptiTips®.

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The Harvard Applier OptiTips® can be sterilized by hot air stream (134°C for 3 minutes). Desinfection with alcohol or 1,5 % glutaraldehyde-containing mixtures for desinfection is possible (max. 1 hour). Use of any other sterilization method will make void the warranty.

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Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental Internationals sole obligation shall be repair or replacement of the Harvard Dental International product.

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Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Bestell-Nr. Artikel
7095200 Harvard Applier OptiTips®

Order-No. Article
7095200 Harvard Applier OptiTips®