Harvard BioCal®-Line

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Harvard BioCal®-Line**Bioactive resin reinforced MTA protective liner, light cure**

Harvard BioCal®-Line is a bioactive alkaline light cure resin reinforced liner with MTA-filler. With the supplied needle tips Harvard BioCal®-Line can be applied very precisely, even in deep cavity preparations. After light cure filling can placed immediately.

Indications

- Liner
- Indirect pulp capping
- Direct pulp capping

Side effects

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Incompatibility with Other Materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor let the material come into contact with materials containing eugenol.

Application

Prepare and dry the cavity.

When used as liner apply first a dental adhesive to ensure a fast adhesion. In case of small scale (pinhead sized) direct and indirect pulp capping an adhesive can be dispensed with.

Screw off the cap of the syringe and substitute it by one of the supplied needle tips (clockwise). Squeeze out the first small drop on a pad to ensure safe handling of Harvard BioCal®-Line.

Apply Harvard BioCal®-Line in one layer (layer thickness ≤ 1 mm). Take care not to apply Harvard BioCal®-Line to the cavity edges and remove any paste sticking to them.

Light cure 40 seconds by using a polymerization unit (wavelength range 400-500 nm) with a light intensity of at least $1000\text{mW}/\text{cm}^2$.

After use discard the needle tip and attach the cap to the syringe again.

Needle tips are for single use only due to hygienic reasons!

If no adhesive has been used up to this step, an adhesive should be applied prior to the application of a composite filling material.

Additional Notes

- Do not use any resin to adjust viscosity of Harvard BioCal®-Line.
- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.

Storage

Store at 4 - 25 °C (39 - 77 °F) protected from light and moisture. Close syringe immediately after use. Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!**Keep away from children!****Caution:**

Federal law restricts the sale of this device to or by the order of a dentist.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7081554	Harvard BioCal®-Line 1 g syringe, 12 needle tips
7095162	Harvard NeedleTips H22 refill bag with 50 needle tips.