Harvard PremiumSil Heavy MaxiMix - Fast

Harvard Dental International
GmbH
Margaretenstr. 2 – 4
15366 Hoppegarten, Germany
Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-
international.de
www.harvard-dental.de

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Instructions for Use EN

Vinylpolysiloxane impression material, super hydrophilic

Harvard PremiumSil Heavy MaxiMix is a high hydrophilic and viscous base impression material consisting of addition curing vinyl silicones. In conjunction with Harvard PremiumSil Medium Fast it can serve as an ideal base for the two phase (one step) impression technique (double mixing technique).

Harvard PremiumSil Heavy MaxiMix shows high hydrophilic properties. This improves the wettability and yields an exact reproduction of subgingival areas and other moist surfaces.

Harvard PremiumSil Heavy MaxiMix meets the requirements of ISO 4823, type 2 (medium-bodied consistency).

Contraindications/side effects

None known

Application

1. Preparation of the cartridge

Harvard Dental International GmbH only recommends for Harvard PremiumSil Heavy MaxiMix the use of mixing cannulas type Harvard Maxi 5:1 Dynamic.

Hold the cartridge with the sealed outlet openings facing upwards. Take the handle of the sealing cap and remove the sealing cap upwards. Avoid excessive force.

Insert the cartridge into the mixing machine (according to manufacturer instructions). Attach a new mixing tip to the cartridge. After the mixing tip is correctly seated (the central hexagonal socket at the lower end of the tip must be properly aligned with the hexagonal drift shaft), place the yellow bayonet ring over the mixing tip and push it down completely. Turn the ring clockwise (1/4 turn) to lock the mixing tip securely in place.

Check whether the hexagonal drift shaft does engage when starting. The spiral in the mixing tip must rotate when starting.

If the used mixing machine offers different mixing speed levels, use always the lowest mixing speed.

Note:

Discard the first 3 cm of impression material extruded from newly filled cartridges. Store used cartridge with fixed used mixing tip. It serves as a cap.

To remove the used mixing tip turn the bayonet ring anti-clockwise and draw it off.

2. Impression taking

Load the auto mixed material onto the tray. Parallel, apply Harvard PremiumSil Medium partially onto the teeth and into the interdental spaces, partially on top of Harvard PremiumSil Heavy MaxiMix.

Insert the tray not later than **1:30 min** (working time at 23°C or 74°F) after extruding Harvard PremiumSil Heavy MaxiMix into the mouth. Maintain position for at least **2:00 minutes**.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F).

The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with standard disinfection solutions.

Notes:

Do not contact the product with latex gloves, they may inhibit the setting. Do not touch teeth with latex gloves.

Do not use with standard (condensation reaction type) silicone impression materials.

Storage

Do not store above 25 °C (77 °F). Do not use after expiry date.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!

Keep the instructions for use for the duration of the application.

Order-No.

7083733

Article

Harvard PremiumSil Heavy MaxiMix, 380ml maximix cartridge, 10 dynamic mixers, 1 bayonet ring
7091400 Harvard Maxi 5:1 Dynamic Refill bag with 50 dynamic mixers and 2 bayonet rings

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