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**Instructions for Use EN**

**Vinylpolysiloxane impression material, super hydrophilic**

Harvard PremiumSil Medium is very versatile. It can be used:

- as a hydrophilic syringeable material for the (one step) two phase impression technique in combination with Harvard PremiumSil Putty Soft or Harvard PremiumSil Heavy
- as a correction material in the (two step) putty/wash technique in combination with Harvard PremiumSil Putty
- as a syringable one phase impression material for crowns, bridges, inlays, partial and complete dentures. It also serves as an impression material for relining, functional impressions, impressions for denture repairs and for impressions using the copper band technique.

Harvard PremiumSil Medium is a light to medium viscosity hydrophilic precision impression material based on addition curing vinyl silicones delivered in automix-cartridges. It can be extruded directly onto the teeth or the tray.

Harvard PremiumSil Medium shows strong hydrophilic properties. Therefore it improves the wettability and yields an exact reproduction of subgingival areas and other moist surfaces.

Harvard PremiumSil Medium is thixotropic. That means it stays in the position in the tray and on the tooth, but under pressure it flows.

Harvard PremiumSil Medium meets the requirements of ISO 4823, type 3 (light-bodied consistency).

**Contraindications/side effects**  
None known

**Application**

**1. Preparation of the automix-cartridge**  
Harvard Dental International GmbH only recommends for Harvard PremiumSil Medium the use of mixing tips type Harvard Auto 1:1 O-Yellow.

Mount the cartridge into the application gun. Remove and dispose the closure-cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved. Install the mixing tip onto the cartridge and turn it 90° clockwise until it locks in position, and extrude needed material.  
For direct extruding onto teeth use Intra Tips yellow.

**Note:** Store used cartridge with fixed used mixing tip. **Do not re-use the cartridge cap.** Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

Leave the used mixing tip on the cartridge. It serves as a cap.

**2. Impression taking**  
Load the auto mixed material onto the tray or directly onto the teeth. Insert the tray not later than **1:10 min** (working time at 23°C or 74°F) after extruding Harvard PremiumSil Medium into the mouth. Maintain position for at least **1:30 minutes**.

**3. Casting**  
The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

**4. Electroplating**  
All impressions can be copper- or silverplated.

**5. Disinfection**  
Impressions can be disinfected with standard disinfection solutions.

**Notes:**  
Do not contact the product with latex gloves, they may inhibit the setting. Do not touch teeth with latex gloves.

Do not use with standard (condensation reaction type) silicone impression materials.

**Storage**  
Do not store above 25 °C (77 °F). Do not use after expiry date.

**Warranty**  
Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

**Limitation of liability**  
Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

**For dental use only!**  
**Store product out of reach of children!**  
Keep the instructions for use for the duration of the application.

**Order-No. Article**  
7083712 Harvard PremiumSil Medium Fast, 2 automix cartridges ea. 50 ml, 12 mixing tips 1:1, 4 Intra Tips  
7091300 Harvard Auto 1:1 O-Yellow, refill bag with 50 mixing tips  
7083619 Harvard Intra Tips yellow refill bag with 50 intra tips.  
7095100 Harvard Dispenser Automix 1:1

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