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Harvard BioPhosphate Dentine HandMix

Bioactive MTA zinc phosphate dentin substitute

Product description

Harvard BioPhosphate Dentine is a MTA modified zinc phosphate cement for dentin replacements restorations. The material has a high pH of 11-12.

Indications/Intended use

- Base for direct restorations (dentin replacement)
- Core build-up
- Indirect pulp capping

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use this product in cases where known allergies to the components exist.

Irritations resulting from direct contact with the pulp cannot be ruled out. In case of an open pulp always apply a thin layer of a pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Interaction with other materials

Avoid direct contact with products containing eugenol since eugenol impairs the setting of the material.

Application

1. Tooth Preparation

Prepare the cavity. Rinse thoroughly with water and dry gently, but do not desiccate.

If the pulp is open, apply a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP).

2. Dispensing and Mixing

The **powder/liquid ratio** is **2.0/1.0**. This can be obtained by mixing **1 level of the supplied scoop of powder and 2 drops of liquid**.

Note:

The spoon dosage is only an approximate reproduction of the nominal mixing ratio.

For accurate dispensing of Harvard BioPhosphate Dentine powder shake the bottle to loosen the powder. Overfill the spoon with the powder, level the powder for using the collar at the top of the bottle and carry it onto the mixing pad. Avoid compressing powder into the spoon with the inside wall of the bottle.

For dispensing of Harvard BioPhosphate Dentine liquid turn the bottle vertically with the tip about 5cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense one drop. If any bubbles are present, lightly tap the bottle with the fingers holding it. **Discard drops** that contain bubbles and are obviously not full-sized.

Use a small spatula to rapidly mix all the cement powder into the liquid.. The mixed cement should be thixotropic and have a smooth consistency and glossy appearance. Total mixing time is **60 seconds**.

Harvard BioPhosphate Dentine is extremely moisture sensitive!
Immediately after use, tightly close both liquid and powder bottles to prevent exposure to moisture.

3. Filling and Finishing

Mix the required amount of Harvard BioPhosphate Dentine and apply it into the prepared cavity by using a suitable placement instrument. Start at the deepest point of the cavity and fill in a single layer. Fill the material into the cavity within **1:30 minutes** (working time from start of mixing at 23 °C or 74°F). Leave enough space for the outer layer.

Net setting time from end of mixing is **3:00 minutes**.

Please see to it, that no air bubbles will be incorporated!

Complete the restauration with a suitable filling material (e.g. Harvard POWERmer Bulk or a light curing composite) according to the instructions of the product used.

Note:

If a light curing composite is used, selective enamel etching with subsequent application of a suitable adhesive is required. This step is not necessary, if Harvard POWERmer Bulk is used.

Note: Higher temperatures will shorten the working time, lower temperatures will prolong the working time.

An overextended working time will cause the loss of adhesion to the dental enamel and the dentine.

4. Additional Notes/Warnings

- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- Irritations of the pulp may occur following the cementing.
- The liquid contains phosphoric acid and is corrosive.
- If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water.
- Keep away from children!

5. Storage

Store at 10 - 25°C (50 - 77°F). **Store protected from moisture.** After first opening use up within 3 months and before the end of expiry date. Do not use after expiry date.

Composition

Powder: MTA, zinc oxide, magnesium oxide

Liquid: buffered phosphoric acid

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed/>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

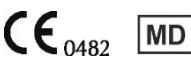
Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No. Article

7071510 5 g powder / 2.5 ml liquid, dosage spoon, mixing pad universal

 Manufacturer	Harvard Dental International GmbH Robert-Bosch-Strasse 2 25335 Elmshorn, Germany	 Made in Germany	Tel: +49 (0) 30/99 28 978-0 Fax: +49 (0) 30/99 28 978-19 info@harvard-dental-international.de www.harvard-dental.de	V01_MDR 2025-03
---	---	--	--	--------------------