## Harvard POWERmer Bulk OptiCaps®

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## Harvard POWERmer Bulk OptiCaps®

**High strength, self adhesive, dual cure resin modified glass ionomer cement for restorations**

### Product description

**Harvard POWERmer Bulk OptiCaps®** is a dual cure resin reinforced self-adhesive glass ionomer restorative material with high strength that is also suitable as an amalgam alternative. Harvard POWERmer Bulk OptiCaps® has good bonding properties to dentine and enamel and does not shrink. With Harvard POWERmer Bulk OptiCaps®, the cavity can be filled in just one layer up to the occlusal surface (Bulk-Fill). No adhesive is required.

Harvard OptiCaps® can quickly and simply be activated and extruded with the Harvard Applier OptiCaps®. Capsule mixing (mixing time 10 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min. Application can be done directly from the capsule.

### Indications/Intended use

- Restorations of cavity classes I, II, III and V
- Core build-up
- Base and liner

### Performance features

The performance features of the product meet the requirements of the intended use.

### Contraindications

Do not use this product in cases where known allergies to the components exist. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

### Patient target group

Persons who are treated during a dental procedure.

### Intended users

This medical device should only be used by a professionally trained dental practitioner.

### Interaction with other materials

Avoid direct contact with products containing eugenol since eugenol impairs the setting of the material.

### Application

#### 1. Tooth Preparation

Prepare the cavity. Rinse thoroughly with water and dry gently, but do not desiccate.

For pulp protection areas close to the pulp should be covered with a thin layer of an alkaline liner (e.g. Harvard BioCal®-Line).

#### 2. Activation and Mixing

The activation and mixing of the OptiCaps® is done in correspondence with the directions of the "Instructions for the activation and mixing of Harvard OptiCaps®".

**Mixing time for Harvard POWERmer Bulk OptiCaps®: 10 seconds**

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent application of the material.

All the cement required should be dispensed from the OptiCaps® within **30 seconds** from the start of mix.

#### 3. Filling and Finishing

Apply the mixed Harvard POWERmer Bulk OptiCaps® directly out of the OptiCaps® into the prepared cavity. Start at the deepest point of the cavity and fill in a single layer up to the desired height.

Please see to it, that no air bubbles will be incorporated!

Fill the material into the cavity within **1:00 minute** (working time from start of mixing at 23 °C or 74°F) and shape the surface with suitable instruments. A matrix can be used to shape the surface.

Cure for **20 seconds** with a visible light-curing device (wavelength range 400–500 nm, light intensity min. 1000 mW/cm<sup>2</sup>). Net setting time without any light is **2:00 – 2:30 minutes**.

Note: Higher temperatures will shorten the working time, lower temperatures will prolong the working time.

An overextended working time will cause the loss of adhesion to the dental enamel and the dentine.

After Harvard POWERmer Bulk OptiCaps® has set, remove matrix where applicable and perform finishing under water spray using standard techniques and polish.

### 4. Additional Notes/Warnings

- Unpolymerized material may have an irritant effect and can lead to sensitization against methacrylates.
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

### 5. Storage

Store at 4 - 25°C (39 - 77°F). Keep the original seal pack closed until use. Do not use after expiry date.

OptiCaps® are for single use only.

Average content per OptiCaps®: 0.5 g

### Composition

Powder: Dental glass, photo initiators

Liquid: polyalkenoate acid, methacrylates

### Disposal

Disposal of the product according to local authority regulations.

### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

### Limitation of Liability


Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

| Order-No. | Article                    |
|-----------|----------------------------|
| 7071212   | 10 OptiCaps® ea. 0.5 g, A2 |
| 7071202   | 50 OptiCaps® ea. 0.5 g, A2 |
| 7092000   | Harvard Applier OptiCaps®  |


### Instructions for the activation and mixing of Harvard OptiCaps®

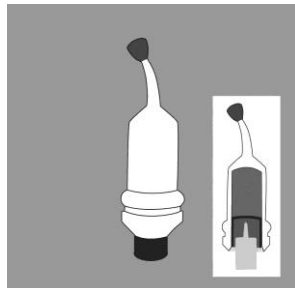Harvard OptiCaps® before activation.


For activation of the Harvard OptiCaps®, press the plunger on a hard and plane surface to the end into the capsule.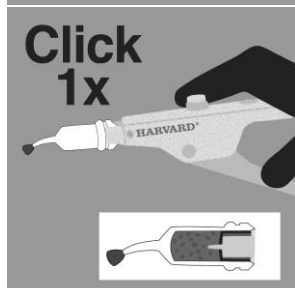


Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps® and click once.

Note: The plunger must be at the same level as the bottom of the capsule.

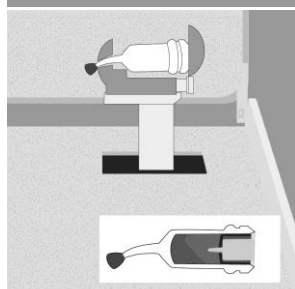
Insert the Harvard OptiCaps® into a mixer, close lid and mix immediately for **10 seconds** at 4,300 oscillations/minute.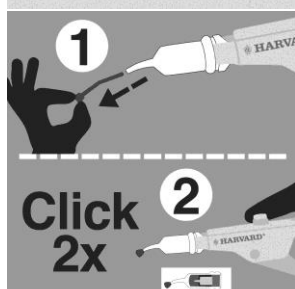

Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps®. Remove the pin from the nozzle. If you leave it, the capsule can burst.

Pull the lever 2 times (2 clicks) to prime the the Harvard OptiCaps®.

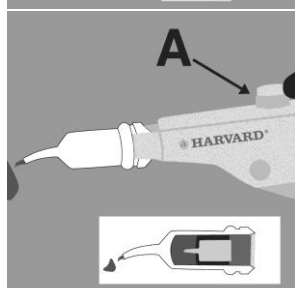Extrude the mixed material on a glass plate or apply directly.

Unlock the Harvard Applier OptiCaps® (press button A) and remove the Harvard OptiCaps®.

The optimal amount of mixed material is guaranteed only with the Harvard Applier OptiCaps® (Order-No. 7092000).

For the selection of a suitable capsule mixer, our sales and marketing colleagues are gladly available to you.

Harvard Dental International GmbH  
Robert-Bosch-Strasse 2  
25335 Elmshorn, Germany

Made in Germany

Tel: +49 (0) 30/99 28 978-0  
Fax: +49 (0) 30/99 28 978-19  
info@harvard-dental-international.de  
www.harvard-dental.de

V02\_MDR  
2025-02