Harvard POWERmer Cem OptiCaps®

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Harvard POWERmer Cem OptiCaps®

High strength, self adhesive, dual cure resin modified glass ionomer cement for luting

Product description

Harvard POWERmer Cem OptiCaps® is a dual cure resin reinforced self-adhesive glass ionomer luting cement. Harvard POWERmer Cem OptiCaps® is translucent, has good bonding properties to dentine and enamel and does not shrink. No adhesive is required.

Harvard OptiCaps® can quickly and simply be activated and extruded with the Harvard Applier OptiCaps®. Capsule mixing (mixing time 10 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min. Application can be done directly from the capsule.

Indications/Intended use

- Luting of crowns and bridges (e.g. zirconia, metal, ceramic)
- Luting of inlays, onlays, veneers
- Luting of posts

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use this product in cases where known allergies to the components exist.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Interaction with other materials

Avoid direct contact with products containing eugenol since eugenol impairs the setting of the material.

Application

1. Preparation

Clean the luting side areas of the restoration according to manufacturer instructions.

2. Activation and Mixing

The activation and mixing of the OptiCaps® is done in correspondence with the directions of the "Instructions for the activation and mixing of Harvard OptiCaps®".

Mixing time for Harvard POWERmer Bulk OptiCaps®: 10 seconds

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent application of the material.

All the cement required should be dispensed from the OptiCaps® within **30 seconds** from the start of mix.

3. Cementing

Apply Harvard POWERmer Cem OptiCaps® directly from the OptiCaps® into the lumens of the crown or bridge (approx. half) resp. to the bonding surface of inlays or onlays in a thin layer. Seat the restoration immediately (**working time 2:00 minutes** from start of mixing at 23°C). Net setting time without any light is **3:00 minutes**.

Remove excess cement at the first setting stage.

Maintain isolation until the set of the cement is verified (**3:00 minutes**). Setting can be optimized with **20 seconds** light cure with a suitable dental light cure unit (wavelength range 400–500 nm, light intensity min. 1000 mW/cm²).

Note:

Higher temperatures will shorten the working time, lower temperatures will prolong the working time.

An overextended working time will cause the loss of adhesion to enamel and dentine.

4. Additional Notes/Warnings

- Unpolymerized material may have an irritant effect and can lead to sensitization against methacrylates.
- Avoid contact with skin, mucous membrane and eyes
- If the material comes into contact with skin, In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

5. Storage

Store at 4 - 25°C (39 - 77°F). Keep the original seal pack closed until use. Do not use after expiry date.

OptiCaps® are for single use only.

Average content per OptiCaps®: 0.5 g

Composition

Powder: Dental glass, photo initiators

Liquid: polyalkenoate acid, methacrylates

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.


Keep the instructions for use for the duration of the application.

Order-No. Article

7071321 10 OptiCaps® ea. 0.5 g, universal
7071320 50 OptiCaps® ea. 0.5 g, universal
7092000 Harvard Applier OptiCaps®


Instructions for the activation and mixing of Harvard OptiCaps®

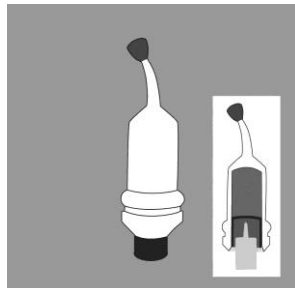Harvard OptiCaps® before activation.


For activation of the Harvard OptiCaps®, press the plunger on a hard and plane surface to the end into the capsule.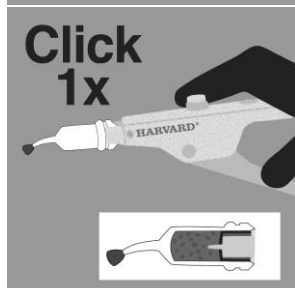


Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps® and click once.

Note: The plunger must be at the same level as the bottom of the capsule.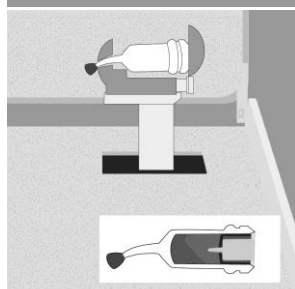


Insert the Harvard OptiCaps® into a mixer, close lid and mix immediately for **10 seconds** at 4,300 oscillations/minute.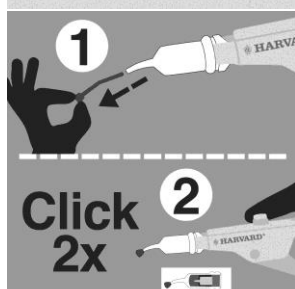

Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps®.

Remove the pin from the nozzle. If you leave it, the capsule can burst.

Pull the lever 2 times (2 clicks) to prime the the Harvard OptiCaps®.

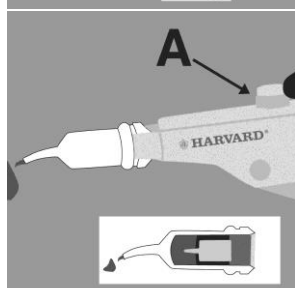Extrude the mixed material on a glass plate or apply directly.

Unlock the Harvard Applier OptiCaps® (press button A) and remove the Harvard OptiCaps®.

The optimal amount of mixed material is guaranteed only with the Harvard Applier OptiCaps® (Order-No. 7092000).

For the selection of a suitable capsule mixer, our sales and marketing colleagues are gladly available to you.

Harvard Dental International GmbH
Robert-Bosch-Strasse 2
25335 Elmshorn, Germany

Made in Germany

Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de

V01_MDR
2025-03