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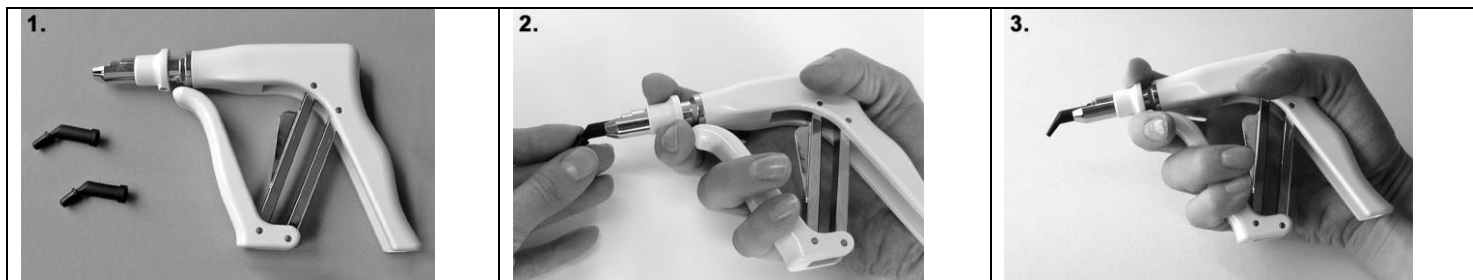
V02_MDR
 2024-12

 Manufacturer	Harvard Dental International GmbH Robert-Bosch-Strasse 2 25335 Elmshorn, Germany	 	Made in Germany

Intended Purpose

The ergonomical designed Harvard Applier OptiTips® (1) is for intraoral application of one-component dental materials (e.g. Light Cure Composites, Retraction paste) in OptiTips®. It is a reusable product.

Indications

Application of materials in OptiTips® (e.g. Light Cure Composites, AppliCord)

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications and side effects

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by professionally trained dental technician users.

General Information

This Dispenser is delivered non-sterile.

Use of the Harvard Applier OptiTips® (see picture 2 and 3)

For inserting or changing of the OptiTips®, the white plastic ring should be pushed back. Now the OptiTips® can be placed easily (2). Then release the ring gently.

Now the material can be easily and reliably applied. Material should easily extrude from the OptiTips®. Move the lever with an even motion (3). **Do not use excessive force!** Excessive force may result in uncontrolled flow of material or cause the OptiTips® to eject. If necessary take a new OptiTips®. After application push the white plastic ring back and remove the OptiTips® from the Harvard Applier OptiTips®.

Warnings/Safety Information

- Avoid fast extrusion of the material from the OptiTips®.
- Do not use excessive force when moving the lever.
- Before each use check the functionality (ease of movement)
- Keep away from children!

Direction for processing according EN ISO 17664

Reprocessing

Disassembly of the Harvard Applier OptiTips® is not necessary.

Warnings

Please follow the country-specific regulations on hygiene standards and cleaning/disinfection of medical devices in dental practices.

Limitations on processing

- Any Harvard Applier OptiTips® showing visible damage or wear must not longer be used.
- End of life of a reusable dispenser is normally determined by wear and damage due to use.
- Use-life of the dispenser is exceeded if visual inspection shows unacceptable deterioration like wear, discoloration, crackles, etc.

Initial treatment at the point of use

- Immediately after use wipe the Harvard Applier OptiTips® with a lint-free cellulose tissue.
- The dispenser should be transported to the area for cleaning and sterilization in a suitable transport box or placed in a solution from detergent and disinfection.

Preparation

Wear suitable gloves, water repellent protective skirt and protective glasses for cleaning and disinfection

Manual cleaning

- Prepare the cleaner according to manufacturer instructions.
- Immerse the Harvard Applier OptiTips® completely in the cleaning solution and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid
- Clean critical parts of the Harvard Applier OptiTips® for 1 Minute with a soft brush.
- Move all moveable parts several times in the cleaning solution.
- Leave the Harvard Applier OptiTips® in the cleaning solution for at least 5 minutes.
- For completely removal of the cleaning solution rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.

Check the Harvard Applier OptiTips® on cleanliness. If necessary, repeat all above steps.

Manual disinfection

Disinfection with alcohol or 1.5 % glutaraldehyd-containing mixtures for disinfection or sterilization is possible (max. 1 hour).

- Prepare the disinfectant to manufacturer instructions.
- Immerse the Harvard Applier OptiTips® completely in the disinfectant and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid.
- Move all moveable parts several times in the cleaning solution.
- For exposure time refer to manufacturer instructions.
- For completely removal of the disinfectant rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.
- Wipe with a lint free tissue or dry with clean compressed air.

Drying

- Allow the Harvard Applier OptiTips® to dry until visibly dry.

Maintenance, Inspection and Testing

Inspect the Harvard Applier OptiTips® under normal lighting:

- Any Harvard Applier OptiTips showing visible damage or wear must not longer be used.
- Repeat manual cleaning and inspection if the Harvard Applier OptiTips® is not visibly clean.

Sterilization

The Harvard Applier OptiTips® can be sterilized by hot steam (134°C/273 °F for 3 minutes).

- Place the Harvard Applier OptiTips® in the sterilizer chamber.
- Start the sterilization program.
- At the end of the sterilization program remove the Harvard Applier OptiTips® and let it cool down.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Warranty of the Harvard Applier OptiTips® will be terminated after 2 years from date of delivery.

Order-No. Article

7095200 Harvard Applier OptiTips®