# Harvard Bite

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## Instructions for Use EN

### Bite registration material based on vinylpolysiloxane with vanilla flavour

#### Product description

**Harvard Bite** is an extremely hard bite registration material with vanilla aroma for fast and precise bite registrations. Harvard Bite material is based on addition curing vinyl silicones. With the automix cartridge Harvard Bite can be applied directly and bubble-free.

Harvard Bite is very suitable for cutting and contouring because of its extremely high final hardness, while not being brittle.

Thanks to its thixotropy, undesired flow of Harvard Bite into dental interstices is prevented and therefore allows an easy removal from the mouth.

Harvard Bite meets the requirements of **ISO 4823, type B** (bite registration materials).

#### Indications/Intended use

- Bite registration and key material for other intraoral recording of jaw relations
- Fixation of bite forks for use with face bows

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

None known

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Application

##### 1. Preparation of the cartridge

Harvard Dental International GmbH recommends for Harvard Bite the use of mixing tips type Harvard Auto 1:1 O-Green only.

Remove the cap of the automix-cartridge and throw it away (**do not use it again!**). Attach a mixing tip. Make sure that the guidance of the automix-cartridge is aligned with that of the mixing tip and turn the tip 90° clockwise until it locks in position. When the application gun is loaded with the prepared cartridge it is ready for application.

#### Note:

Discard the initial extrusion from the mixing tip (about the size of a pea). The following mix is perfect.

Leave the used mixing tip on the cartridge. It serves as a cap.

##### 2. Working time

Working time at 23° C: **20 seconds**  
Recommended time in mouth: **45 seconds**

##### 3. Bite registration

The automatically mixed material is applied directly onto the occlusal areas. Within **20 seconds** after the beginning of the extruding (working or application time at 23 °C or 74 °F) allow the patient to adopt the desired occlusion. The recommended time in mouth is about **45 seconds**.

##### 4. Articulator

Harvard Bite allows a visual control during fixing of the gypsum model in the articulator. Registrates made from Harvard Bite can be stored over months without shrinkage.

##### 5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones. Refer to the manufacturer instructions.

#### Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

#### Additional notes:

- Do not use latex gloves because they may inhibit the polymerization of vinyl silicones. Do not touch prepared tooth with latex gloves.
- Keep away from children!

#### Disposal

Disposal of the product according to local authority regulations.

#### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

#### Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

#### Caution:

**Federal law restricts the sale of this device to or by the order of a dentist.**

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