# Harvard Cement OptiCaps®

V02\_MDR  
2025-06

Harvard Dental International GmbH  
Robert-Bosch-Strasse 2  
25335 Elmshorn, Germany

Tel: +49 (0) 30/99 28 978-0  
Fax: +49 (0) 30/99 28 978-19  
info@harvard-dental-international.de  
www.harvard-dental.de

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### Zinkphosphat-Zement

#### Produktbeschreibung

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### Zinc phosphate cement

#### Product description

Harvard Cement OptiCaps® is classical cement based on zinc oxide and phosphoric acid.

#### Indications/Intended use

- Cementing of crowns and multi-unit bridges (made of precious metal, non-precious metal, metal-ceramics, zirconium dioxide, aluminium oxide and lithium disilicate ceramics) as well as gold inlays
- Cementing of implant-based crowns and multi-unit bridges

#### Composition

Powder contains: zinc oxide, magnesium oxide, pigments  
Liquid contains: phosphoric acid.

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Activation and mixing

See "Instructions for the activation and mixing of Harvard OptiCaps®".  
Harvard Cement OptiCaps® are easily activated and the content of the capsule is easily ejected with the Harvard Applier OptiCaps®. Mixing of Harvard Cement OptiCaps® is achieved by a high frequency mixer with about 4,300 oscillations/minute. The material can be applied directly from the OptiCaps®.

Mixing time for the Harvard Cement OptiCaps® is **10 seconds**.

#### Attention:

Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting which may impair or prevent application of the material. The cement must be extruded within 20 seconds after the end of mix.

Harvard Cement OptiCaps® sticks to metal instruments and should be washed off with cold water prior to setting.

#### Application

Apply the mixed cement to the prepared bonding surface of the restoration. Seat the restoration immediately.

**Working time** from start of mixing at 23 °C/73 °F:

**1:30 minutes  
< 8 minutes**

**Net setting time** at 37 °C:

#### Special information

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
- Irritations of the pulp may occur following the cementing.
- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- The liquid contains phosphoric acid and is corrosive. In case of accidental eye contact immediately rinse with large quantities of water.
- Keep away from children!

**Useable volume of mixed cement per OptiCaps®:**

0,13 to 0,14 ml.

#### Storage

Store at 2 – 28 °C (36 °F – 82 °F).  
Do not use after expiry date.

Harvard Cement OptiCaps® are for single use only.

#### Disposal

Disposal of the product according to local authority regulations.

#### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

#### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

#### Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

#### Harvard Cement conforms to the ISO 9917-1.

Keep the instructions for use for the duration of the application.

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