HARVARD[®]

Harvard Dental International GmbH

Robert-Bosch-Strasse 2

25335 Elmshorn, Germany

Tel: +49 (0) 30/99 28 978-0

Fax: +49 (0) 30/99 28 978-19

info@harvard-dental-international.de

www.harvard-dental.de

Harvard Cement normal härtend normal setting

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Instruction for use for the dentist **EN**

Product description

Harvard Cement normal setting is classical cement based on zinc oxide and phosphoric acid.

Composition:
1 g powder contains:
860 mg zinc oxide, 86 mg magnesium oxide;
1 g liquid contains:
560 mg o-phosphoric acid.

Indications/intended use

Fixation of crowns and multiple bridges
- also on implants
- also for zirconium dioxide, aluminium oxide and lithium disilicate ceramics,
luting of inlays, cavity linings, temporary fillings in posterior teeth, reconstruction of the core, orthodontic works.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Dispensing Instructions:

Fixation consistency:
1.5 g powder : 1.0 g liquid
Cavity lining consistency:
2.1 g powder : 1.0 g liquid
Dispensing and testing climate for testing in accordance with DIN EN ISO 9917: Mix quantities indicated at 23 ± 1°C and a relative humidity of 50 ± 10%.

Processing Recommendations:
A clean, dry glass plate at ca. 20°C (68 °F) and a nondiscolouring spatula are recommended for mixing.

Mixing:

In order to achieve a homogenous consistency, divide the measured quantity of powder into 4 portions (1/2 1/4 1/8 1/8). Beginning with the smallest quantity, stroke the powder into the measured liquid within 90 seconds. The filling consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste. The fixation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste. Discard remaining powder or impure liquid.

Mixing Time: 90 seconds.
Processing Time from start of mixing (at 23 °C, 72 °F):
For fixation: 3 minutes
For cavity lining: 2 minutes
Setting Time from end of mixing:
5:00 – 8:00 minutes

Special Information/Warnings:

- The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.
- Irritations of the pulp may occur following the cementing and in the case of the mixture being used for cavity lining.
- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- Close containers immediately after removing material.
- The liquid contains phosphoric acid and is corrosive. In the event of accidental eye.
- Keep away from children!

Shades:

1 (white), 2 (bluish white), 3 (yellowish white), 4 (light yellow), 5 (yellow)
Package Sizes:
Powder: 100 g and 35 g
Liquid: 40 ml and 15 ml

Harvard Cement conforms to the DIN EN ISO 9917.
Keep the instructions for use for the duration of the application.

Storage
Store **Harvard Cement** at a dry place at room temperature (2 – 28°C/36-82 °F). Do not use after the expiration date.

Disposal
Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – https://ec.europa.eu/tools/eudamed).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH’s sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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