Harvard IonoGlas Cem Extra

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Harvard IonoGlas Cem Extra

Classical Glass Ionomer Luting Cement

Product description

Harvard IonoGlas Cem Extra is a new glassionomer luting cement with improved physical properties.

Beside its high fluoride release and the excellent biocompatibility as glassionomer cement Harvard IonoGlas Cem Extra has also good chemical bonding to dentine and enamel and to the restoration and a tight seal at the dentinal margins. Because of its radiopacity it ensures easy postoperative diagnosis.

Indications/Intended use

- cementing of crowns and bridges (all types: metal, ceramic fused to metal, ceramic and zirconia)
- cementation of metal inlays and onlays
- liner under composite fillings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

- Pulp capping
- Allergic reactions to glass ionomer cements

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation

Prepare the tooth to be treated in the accustomed manner. Clean the prepared tooth with pumice and water. Rinse thoroughly and dry, but **do not** desiccate.

By using a cotton pellet apply a dentin conditioner to remove the smear layer. Refer to the instructions of the product used. Rinse the dentin conditioner with water and dry it in an airstream, but **do not** desiccate.

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

2. Dosing of powder and liquid

The powder/liquid ratio to achieve a suitable consistency is 1.8 / 1.0. Mix 1 level spoon (blue) of powder and 2 drops of liquid.

Note:

The spoon dosage is only an approximate reproduction of the nominal mixing ratio.

Shake the bottle to loosen the powder. Overfill the spoon with the powder, level the powder for using the collar at the top of the bottle and carry it onto the mixing pad provided. Avoid compressing powder into the spoon with the inside wall of the bottle.

Turn the liquid bottle vertically with the tip about 5cm above the mixing pad. Steady your hand and squeeze the bottle gently to dispense one drop. If any bubbles are present, lightly tap the bottle with the fingers holding it. **Discard drops** that are obviously not full-sized.

After use, tightly close both liquid and powder bottles to prevent exposure to moisture.

3. Mixing of powder and liquid

Use a small spatula to rapidly mix all the cement powder into the liquid. Usually one scoop powder/ two drops liquid should provide for sufficient amount of mixed cement. The mixed cement should be thixotropic and have a smooth consistency and glossy appearance. Total mixing time is **30 seconds**.

4. Cementation Technique

prepared bonding surface of the restoration with 1 mm of cement (Do not overfill!) and seat immediately within the working time. Seat the restoration by using steady, even pressure. Remove excess cement at the first setting stage.

Maintain isolation until the set of the cement is verified.

Working time from start of mix at 23°C (74°F): **1:30 minutes**
Net setting time at 37°C (99°F): **5:00 minutes**

Notes

Higher temperatures will shorten the working time, lower temperatures will prolong the working time.
 An overextended working time reduces adhesion to enamel and dentine.

5. Additional Notes/Warnings

- Do not use the material with patients who show an allergy to the material. In case of allergic reactions immediately stop the application, and advise the patient to consult a physician. An operator, who has a history of allergy to glass ionomer cements, should not handle with Harvard IonoGlas Cem Extra.
- Do not allow the cement mixture to contact the oral tissues or skin. In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water.
- Avoid eye contact of the cement mixture. In case of contact, immediately flush with water and seek medical treatment.
- Do not mix the powder or liquid of Harvard IonoGlas Cem Extra with any other glass ionomer product.
- Shade: universal
- Keep away from children!

Storage

Storage temperature 4-25°C (39-77°F). Store protected from moisture. Do not use after expiry date.

Composition

Dental glass, polyalkenoate acids.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed/>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard IonoGlas Cem Extra conforms to the DIN EN ISO 9917-1.

Keep the instructions for use for the duration of the application.

Order-No. Article

7042115 15 g powder, shade Universal / 10 ml liquid,
dosage spoon, mixing pad