Harvard Ionoglas Cem Extra OptiCaps®

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Harvard Ionoglas Cem Extra OptiCaps®

Esthetic Glass Ionomer Cement for Luting and Lining

Product description

Harvard Ionoglas Cem Extra OptiCaps® is an esthetic glass ionomer luting and lining cement in OptiCaps®.

Beside its high fluoride release and the excellent biocompatibility as glassionomer cement Harvard Ionoglas Cem Extra has also good chemical bonding to dentine and enamel and to the restoration and a tight seal at the dentinal margins. Because of its radiopacity it ensures easy postoperative diagnosis.

Harvard Ionoglas Cem Extra OptiCaps® can quickly and simply be activated and extruded with the Harvard Applier OptiCaps®. Capsule mixing (mixing time 10 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/min. Application can be done directly from the capsule.

Indications/Intended use

- cementing of crowns and bridges (all types: metal, ceramic fused to metal, ceramic and zirconia)
- cementation of metal inlays and onlays
- liner under composite fillings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

- Pulp capping
- Allergic reactions to glass ionomer cements

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation

Prepare the tooth to be treated in the accustomed manner. Clean the cavity preparation with pumice and water. Rinse thoroughly and dry, but **do not** desiccate.

By using a cotton pellet apply a dentin conditioner to remove the smear layer. Refer to the instructions of the product used. Rinse the dentin conditioner with water and dry it in an airstream, but **do not** desiccate.

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

2. Mixing of Harvard Ionoglas Cem Extra OptiCaps®

The activation and mixing of the OptiCaps® is done in correspondence with the directions of the "Instructions for the activation and mixing of Harvard Ionoglas Cem Extra OptiCaps®".

Mixing time for Harvard Ionoglas Cem Extra OptiCaps® : 10 seconds

Avoid lag-times between the processes of activation, mixing and application, as the material is in the process of setting and time-lags may impair or prevent application of the material.

Hence, all the cement required should be dispensed from the Harvard OptiCaps® within **15 seconds** after end of mix.

Harvard Ionoglas Cem Extra sticks to metal instruments and should be washed off with cold water prior to setting.

3. Cementing

Apply the mixed Harvard Ionoglas Cem Extra directly onto the prepared bonding surface of the restoration and seat immediately within the working time. Seat the restoration by using steady, even pressure.

Remove excess cement at the first setting stage.

Maintain isolation until the set of the cement is verified (approximately 5:00 min).

Working time from start of mix at 23°C (74°F): **1:30 minutes**
Net setting time at 37°C (99°F): **5:00 minutes**

Notes

Higher temperatures will shorten the working time, lower temperatures will prolong the working time.

An overextended working time reduces adhesion to enamel and dentine.

Instruct the patient not to expose the restoration to any pressure for one hour.

4. Additional Notes/Warnings

- Do not use the material with patients who show an allergy to the material. In case of allergic reactions immediately stop the application, and advise the patient to consult a physician. An operator, who has a history of allergy to glass ionomer cements, should not handle with Harvard Ionoglas Cem Extra.
- Do not allow the cement mixture to contact the oral tissues or skin. In case of contact, remove the material with absorbent cotton soaked in alcohol and rinse with water.
- Avoid eye contact of the cement mixture. In case of contact, immediately flush with water and seek medical treatment.
- Shade: universal
- Keep away from children!

Storage

Store in the original seal pack at 4-25°C (39-77°F). Store protected from moisture. Do not use after expiry date.

OptiCaps® are for single use only.

Composition

Dental glass, polyalkenoate acids.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

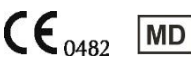
Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.


Harvard Ionoglas Cem Extra conforms to the DIN EN ISO 9917-1.

Keep the instructions for use for the duration of the application.


Order-No. Article

7042250 50 OptiCaps® ea. 0,4 g, Shade universal
 7092000 Harvard Applier OptiCaps®


 Manufacturer	Harvard Dental International GmbH Robert-Bosch-Strasse 2 25335 Elmshorn, Germany	 Made in Germany	Tel: +49 (0) 30/99 28 978-0 Fax: +49 (0) 30/99 28 978-19 info@harvard-dental-international.de www.harvard-dental.de	V02_MDR 2025-10
---	---	--	--	--------------------

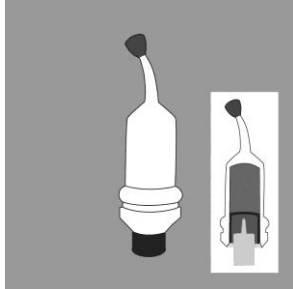Instructions for the activation and mixing of Harvard OptiCaps®

Harvard OptiCaps® before activation.

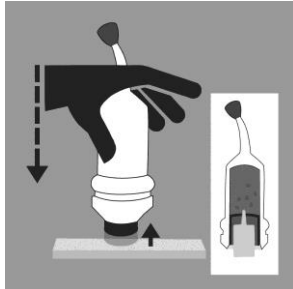
For activation of the Harvard OptiCaps®, press the plunger on a hard and plane surface to the end into the capsule.

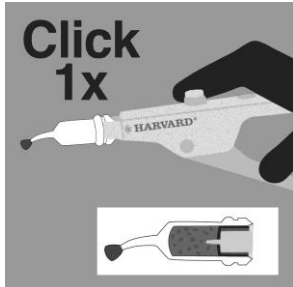
Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps® and click once.

Note: The plunger must be at the same level as the bottom of the capsule.

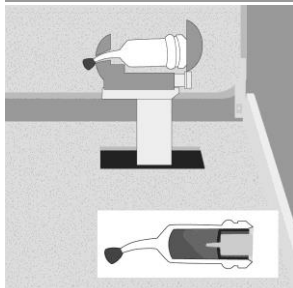
Insert the Harvard OptiCaps® into a mixer, close lid and mix immediately for **10 seconds** at 4,300 oscillations/minute.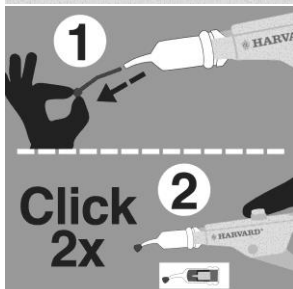

Insert the Harvard OptiCaps® into the Harvard Applier OptiCaps®. Remove the pin from the nozzle. If you leave it, the capsule can burst.

Pull the lever 2 times (2 clicks) to prime the the Harvard OptiCaps®.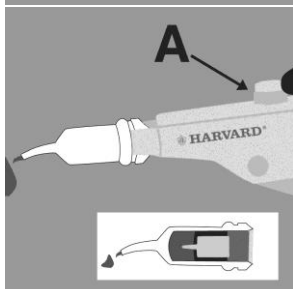

Extrude the mixed material on a glass plate or apply directly.

Unlock the Harvard Applier OptiCaps® (press button A) and remove the Harvard OptiCaps®.

The optimal amount of mixed material is guaranteed only with the Harvard Applier OptiCaps® (Order-No. 7092000).

For the selection of a suitable capsule mixer, our sales and marketing colleagues are gladly available to you.