# Harvard Polycarboxylat Cement

  
 Made in Germany

**Harvard Dental International GmbH**  
 Robert-Bosch-Strasse 2  
 25335 Elmshorn, Germany

 Tel: +49 (0) 30/99 28 978-0  
 Fax: +49 (0) 30/99 28 978-19  
 info@harvard-dental-international.de  
 www.harvard-dental-.de

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### Zink-Polycarboxylat-Zement

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## Instructions for Use EN

### Zinc polycarboxylate cement

**Product description**  
 Harvard Polycarboxylat Cement is a zinc polycarboxylate cement based on zinc oxide and polyacrylic acid.

**Composition:**  
 1 g powder contains:  
 890 mg zinc oxide, 89 mg magnesium oxide  
 1 g liquid contains:  
 400 mg polyacrylic acid

**Indications/intended use**  
 Fixation of crowns and inlays, cavity linings

**Performance features**  
 The performance features of the product meet the requirements of the intended use.

**Contraindications**  
 The mixed cement should never be applied directly onto dentine near to pulp or onto exposed pulp.

**Patient target group**  
 Persons who are treated during a dental procedure.

**Intended users**  
 This medical device should only be used by a professionally trained dental practitioner.

**Dispensing Instructions**  
 Fixation consistency:  
 2,9 g powder : 1,0 g liquid  
 Cavity lining consistency:  
 3,6 g powder : 1,0 g liquid  
 Dispensing and testing climate for testing in accordance with DIN EN ISO 9917:  
 Mix quantities indicated at 23 ± 1°C and a relative humidity of 50 ± 10%.

**Processing Recommendations:**  
 A clean, dry glass plate at approx. 20 °C (68 °F) and a non-discoloring spatula are recommended for mixing.

**Mixing:**  
 Divide the measured quantity of powder into 3 portions (1/2 1/4 1/4). Beginning with the largest quantity, stroke the powder into the measured liquid within 60 seconds. The mixture should appear thicker as usual for zinc phosphate cement, yet it is just as easy to work. The fixation consistency has been reached when the peak formed on lifting the spatula slowly falls back into the paste. The filling consistency has been reached when on lifting the spatula, the peak drawn out forms a small hook and does not sink back into the paste. Discard remaining powder or impure liquids.

**Mixing Time:** 60 seconds

**Processing Time from start of mixing (at 23 °C, 72 °F):**  
 For fixation: 2:00 minutes  
 For cavity linings: 1:30 minutes

**Setting Time from end of mixing:**  
 2:30 – 6:00 minutes

### Special Informations/Warning

- Apply the thinnest possible layer for fixations.
- The cement must be applied to the dry tooth surface and must set without contact to moisture.
- Remove excess material from neighbouring teeth, facets etc. immediately.
- Do not expose liquid to sunlight and frost.
- Close containers immediately after removing material.
- Clean instruments and glass plate with water before the cement sets.
- Zinc polycarboxylate cement can be dissolved using a cement-solvent after setting.
- Keep away from children!

**Storage**  
 Store at 2 – 28 °C (36 – 82 °F). Do not use after expiry date.

**Disposal**  
 Disposal of the product according to local authority regulations.

**Reporting obligation**  
 Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

**Note**  
 The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

**Warranty**  
 Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible

for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International product.

### Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability. Harvard Polycarboxylat Cement meets the requirements of DIN EN ISO 9917. Keep the instructions for use for the duration of the application.

Order-No.	Article
7031503	Single powder, 35 g
7031504	3 yellowish-white 4 light yellow
7031203	Clinic powder, 100 g
7031204	3 yellowish-white 4 light yellow
7031600	Liquid
7031300	Single liquid, 15 ml Clinic liquid, 40 ml

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