Harvard PremiumSil Heavy

Harvard Dental International GmbH
Robert-Bosch-Strasse 2
25335 Elmshorn, Germany

Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de

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Instructions for Use EN

Vinylpolysiloxane impression material, super hydrophilic

Product description

Harvard PremiumSil Heavy is a high hydrophilic and viscous base impression material consisting of addition curing vinyl silicones. In conjunction with Harvard PremiumSil Medium it can serve as an ideal base for the two phase (one step) impression technique (double mix technique).

Harvard PremiumSil Heavy shows high hydrophilic properties. Thus improves the wettability and yields an exact reproduction of subgingival areas and other moist surfaces.

Harvard PremiumSil Heavy meets the requirements of ISO 4823, type 2 (medium-bodied consistency).

Indications/Intended use

- Double-mix technique (1 step/2 phase) with e.g. Harvard PremiumSil Medium.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation of the automix-cartridge

Harvard Dental International GmbH only recommends for Harvard PremiumSil Heavy the use of mixing tips type Harvard Auto 1:1 O-Green.

Mount the cartridge into the application gun. Remove and dispose the closure-cap. For exact flow control extrude slightly material until uniform flow from both orifices is achieved.

Install the mixing tip onto the cartridge and turn it 90° clockwise until it locks in position. and extrude needed material.

Note:

Store used cartridge with fixed used mixing tip. **Do not re-use the cartridge cap.** Closing the cartridge by turning on the cap between use could cause cross contamination of components and thus premature setting.

Leave the used mixing tip on the cartridge. It serves as a cap.

2. Impression taking

Load the auto mixed material onto the tray. Parallel, apply Harvard PremiumSil Medium partially onto the teeth and into the interdental spaces, partially on top of Harvard PremiumSil Heavy. Insert the tray not later than **1:45 min** (working time at 23°C or 74°F) after extruding Harvard PremiumSil Heavy into the mouth. Maintain position for at least **2:30 minutes**.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones. Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- When using the double-mix technique (1 step/2 phase), make sure that both materials have the same setting time in mouth. The working time is determined by the material with the shorter working time.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the

product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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