Harvard PremiumSil Putty Fast

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DE

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Harvard PremiumSil Putty Fast, 2 Dosen à 600g, 2 Löffel

Instructions for Use EN

Vinylpolysiloxane impression material, Putty, kneadable

Product description

Harvard PremiumSil Putty is used as a base impression material for the putty/wash impression technique in combination with Harvard PremiumSil Light or Harvard PremiumSil Medium.

Harvard PremiumSil Putty is a kneadable precision impression material based on addition curing vinyl silicone. Harvard PremiumSil Putty is easy to mix and not sticky.

Harvard PremiumSil Putty meets the requirements of ISO 4823, type 0 (putty consistency).

Indications/Intended use

- Correction impression technique (2 step/2 phase) with e.g. Harvard PremiumSil Light or Harvard PremiumSil Medium

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Mixing

Perforated or non-perforated metal or plastic trays may be used with Harvard PremiumSil Putty. In order to improve adhesion of Harvard PremiumSil Putty to the tray, a silicone adhesive (e.g. Harvard TraySive) should be applied according to the corresponding instructions.

Use equal parts (volume or weight) of base paste and catalyst paste. Take the base

paste with the dark measuring spoon and the catalyst paste with the white measuring spoon. Knead until the color is homogeneous (at least **30 seconds**).

2. Impression taking

Load the ready kneaded material into the prepared tray. Insert the loaded tray in the mouth not later than **50 seconds** (working time at 23°C or 74°F) after start of mix. Maintain position for at least **1:45 minutes**.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones. Refer to the manufacturer instructions.

Storage

Store at 4-25 °C (39-77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not interchange lits of pots. Do not interchange measuring spoons.
- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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