Harvard Restore System Kit Plus

Gebrauchsanweisung		DE
 		
0482		
Made in Germany		
		
Harvard Dental International GmbH Robert-Bosch-Strasse 2 25335 Elmshorn, Germany		
Tel: +49 (0) 30/99 28 978-0 Fax: +49 (0) 30/99 28 978-19 info@harvard-dental-international.de www.harvard-dental.de		
V02_MDR 2024-11		

Produktbeschreibung

Harvard Restore System Kit:

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Product description

Harvard Restore System Kit:

- **Harvard Restore** is a light cure microhybrid composite for anterior and posterior restorations. It is based on BIS-GMA-resin and inorganic filler particles of 0.05-1.5 µm. The total filler content is 81 % by weight and 65 % by volume. Radiopacity of Harvard Restore is 200 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).
- **Harvard RestoreFlow** is a visible light cure easy flowable microhybrid composite for anterior and limited posterior restorations and for sealing. Harvard RestoreFlow is based methacrylate-resin and inorganic filler particles of 0.05-1 µm. The filler content: is 62 % by weight and 47 % by volume. Radiopacity of Harvard RestoreFlow is 150 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).
- **Harvard Self-Bond** is a simple to use self-etching light curing one component adhesive. It ensures a strong bonding of composites to enamel and dentin. Harvard Self-Bond is compatible with all current brands of visible light cure methacrylate based composite restorative materials.

Indications/ Intended use

Restorations of all cavity classes

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.
Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor let the composite allow coming into contact with materials containing eugenol.

Application

1. Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.
Ascertain the tooth shade while teeth are still moist and select the appropriate Harvard Restore shade.
Harvard Restore is shaded according to VITA®-Shades.

2. Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction.
For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

3. Application of Harvard Self-Bond

Apply Harvard Self-Bond with a brush onto the **slightly moist enamel and dentin surfaces** for at least 15 seconds with agitation.

The material should build a homogeneous layer. Air thin gently for 5-10 seconds to remove the volatile components and to disperse the adhesive. Take care, that the adhesive is not removed from the tooth surface by a strong air stream.
Do not touch the treated surface!
Then light cure with a suitable polymerization unit (wavelength range 400–500 nm, light intensity of at least 500mW/cm²) for 20 seconds.

4. Application and curing of Harvard Restore

Apply Harvard Restore in the selected shade. Place it by using suitable instruments. Harvard Restore is easy to model. Transparent matrix strips may be used.
In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement (e.g. Harvard Ionoglas Fill LC) or a light cure flowable composite (e.g. Harvard RestoreFlow), followed by the application of Harvard Restore.
In case of small cavities Harvard Restore can be applied directly.
For an optimum result apply Harvard Restore in **layers of max. 2 mm** (Harvard Restore) resp. **max. 1 mm** (Harvard RestoreFlow). Light cure each increment separately.

By using a polymerization unit (wavelength range 400 – 500 nm) with a **light intensity of at least 1000mW/cm²**, cure each increment as follows:
Lighter shades (A1, A2, A3) **20 seconds**
Darker shades (A3.5) **30 seconds**
Hold the light emission window as close as possible to the filling material.

5. Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs.
Polish to high gloss with polishing discs or polishing brushes.

6. Additional informations/Warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized Harvard Restore and Harvard Self-Bond may have an irritant effect and can lead to sensitization against methacrylates.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

Composition

Harvard Restore, Harvard Restore Flow:
Dimethacrylates, dental glass, silicon dioxide, photo initiators.

Harvard Self-Bond:

Hydrophilic methacrylates, MDP, ethanol, photo initiators.

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.
Close syringe and bottle immediately after use to avoid exposure to light.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

For dental use only!

Store product out of reach of children!

Harvard Restore and Harvard RestoreFlow conform to the ISO 4049, type 1, class 2, group 1.

Keep the instructions for use for the duration of the application.

Order-No.

7083217

Article

Harvard Restore System Kit
4 syringe ea. 4g Harvard Restore
shade A1, A2, A3, A3.5
3 ml syringe Harvard RestoreFlow shade A2
5 ml bottle Harvard Self-Bond
5 needle tips, 50 x Microbrush®,
5 mixing wells

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