Harvard TransMatrix

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Harvard TransMatrix

Transparent matrix and bite registration material based on vinylpolysiloxane

Product description

Harvard TransMatrix is a transparent matrix material based on addition curing vinyl silicones. For this purpose make an impression of the tooth surface and the neighboring teeth before beginning the preparation. This impression builds a tailor made transparent matrix for the subsequent restoration (e.g. veneers) with light curing composites.

Harvard TransMatrix is also suitable as a transparent bite registration material for fast and precise bite registration. Because of the transparency the occlusion can be visually controlled.

With the AUTOMIX cartridge Harvard TransMatrix can be applied free of bubbles directly onto the teeth.

Indications/Intended use

- Matrix material
- Bite registration material

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Preparation of automix-cartridge system

Harvard Dental International GmbH recommends for Harvard TransMatrix the use of mixing tips type Harvard Auto 1:1 O-Green only.

Remove the cap of the automix-cartridge and throw it away (**do not use it again!**). Attach a mixing tip. Make sure that the guidance of the automix-cartridge is aligned with that of the mixing tip and turn the tip 90° clockwise until it locks in position. When the application gun is loaded with the prepared cartridge it is ready for application.

Note:

Discard the initial extrusion from the mixing tip (about the size of a pea). The following mix is perfect.

Leave the used mixing tip on the cartridge. It serves as a cap.

Application

Matrix material

Apply the automatically mixed material onto the area around the surface you want to restore. For better light penetration smooth the surface (e.g. place a mylar strip on the matrix). The final thickness of the matrix should be about 5 mm.

Harvard TransMatrix has a working time of **25 seconds** (at 23 °C or 74 °F) from start of mixing. After a setting time of about **1:20 minutes** remove the matrix from the mouth.

Prepare the tooth as usual and condition the surface (e.g. with Harvard InterLock ONE). Apply a light curing composite (e.g. Harvard UltraFill) according to the manufacturer instructions with slight excess directly onto the tooth or into the matrix. Reposition the matrix and light cure the composite through the matrix. Remove the matrix and light cure again according to the instructions of the applied composite to guarantee complete polymerization. Remove excess material and check the occlusion.

If the composite (e.g. in case of deep cavities) is applied in layers,

apply the final layer with slight excess, reposition the matrix and light cure.

Bite registration

Apply the automatically mixed material directly onto the occlusal surface. Within **25 seconds** after the beginning of the extruding (working time at 23 °C or 74 °F) bring the bite into the desired intercuspitation. The recommended time in mouth is about **1:20 minutes**.

The transparency of Harvard TransMatrix allows a visual control also during fixing of the gypsum model in the articulator.

Disinfection

Disinfection with disinfection solutions specific for materials based on addition curing silicones is possible.

Refer to the manufacturer instructions.

Storage

Store at 4-25 °C (39-77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use latex gloves because they may inhibit the polymerization of vinyl silicones. Do not touch prepared tooth with latex gloves.
- Matrixes and registrates made from Harvard TransMatrix can be stored over months without shrinking.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Caution:

Federal law restricts the sale of this device to or by the order of a dentist.

Keep the instructions for use for the duration of the application.

Order-No. Article

7083798	Harvard TransMatrix 2 x 50 ml automix-cartridge, Shade transparent, 12 mixing tips
7096000	Harvard Auto 1:1 O-Green, refill bag with 50 mixing tips
7095100	Harvard Dispenser Automix 1:1