Harvard TraySive

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Produktbeschreibung

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Tray adhesive

Product description

Harvard TraySive is a universal tray adhesive for silicones based impression materials.

Harvard TraySive ensures a reliable adhesion of Harvard PremiumSil impression material to the impression tray and und thus helps to avoid uncontrollable deformation during impression taking.

Harvard TraySive achieves a secure adhesion between Harvard PremiumSil impression materials and metal- or methacrylate trays.

Indications/Intended use

Adhesive for impression trays

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Contraindications are not known. In case of known hypersensitivity to methacrylates avoid skin contact.

Patient target group

Persons who are treated during a dental procedure.

Intended users

Only for professional trained dental/dental technician users.

Application

Clean the impression tray and dry it thoroughly. Apply a thin layer of Harvard TraySive to the tray using the applicator brush and allow to dry at least **3 minutes**.

Then load the tray und take the impression.

Note:

Harvard TraySive does not always secure sufficient adhesion to trays made of other plastic materials or individual trays made of base plates. A polyether adhesive must be used for polyether impression materials.

Cleaning from the tray

Harvard TraySive can be removed with acetone from the metal impression tray.

Storage information

Do not store above 25 °C (77 °F). Do not use after expiry date.

Important notes

- Avoid contact with eyes and skin. In case of contact wash off with water. Do not inhale vapors!
- Harvard TraySive should only be used in well ventilated rooms.
- Close bottles immediately after use.
- Side effects are not known do date. The residual risk of possible hypersensitivity to individual components of the material cannot be excluded.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental Internationals sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Caution:

Federal law restricts the sale of this device to or by the order of a dentist.

Keep the instructions for use for the duration of the application.

Order-No. Article

7083753 Harvard TraySive, 10 ml bottle