Harvard Applier OptiCaps®

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Harvard Applier OptiCaps®

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- Harvard Cement OptiCaps®
- Harvard MTA OptiCaps® (alle Typen)
- Harvard Ionoglas Cem Extra OptiCaps®
- Harvard Ionoglas Fill Extra OptiCaps®
- Harvard Ionoresin Cem Extra OptiCaps®
- Harvard Ionoresin Fill Extra OptiCaps®

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Intended Purpose

The ergonomic designed Harvard Applier OptiCaps® is for intraoral application of two component dental materials (e.g. MTA cements, glass ionomer cements) in Harvard OptiCaps®. It is a reusable product.

Indications

Application of materials in Harvard OptiCaps® (e.g. MTA cements, glass ionomer cements)

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications and side effects

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by professionally trained dental dental technician users.

General Information

This applicator is delivered non-sterile. Keep the Harvard Applier OptiCaps® in a dry place to prevent from corrosion.

Use of the Harvard Applier OptiCaps® (see Instruction for activating and mixing Harvard OptiCaps®)

Warnings/Safety Information

- Avoid fast extrusion of the material from the Harvard OptiCaps®.
- Do not use excessive force when moving the lever.
- Before each use check the functionality (ease of movement)
- Keep away from children!

Direction for processing according EN ISO 17664

Reprocessing

Disassembly of the Harvard Applier OptiCaps® is not necessary.

Warnings

Please follow the country-specific regulations on hygiene standards and cleaning/disinfection of medical devices in dental practices.

Limitations on processing

- Any Harvard Applier OptiCaps® showing visible damage or wear must not longer be used.
- End of life of a reusable dispenser is normally determined by wear and damage due to use.
- Use-life of the dispenser is exceeded if visual inspection shows unacceptable deterioration like wear, discoloration, crackles, etc.

Initial treatment at the point of use

- Remaining cement should be removed immediately.
- Immediately after use wipe the Harvard Applier OptiCaps® with a lint-free cellulose tissue.
- The dispenser should be transported to the area for cleaning and sterilization in a suitable transport box or placed in a solution from detergent and disinfection.

Preparation

Wear suitable gloves, water repellent protective skirt and protective glasses for cleaning and disinfection.

Note: It is recommended to use a disinfectant solution which contains rust inhibitor.

Manual cleaning

- Prepare the cleaner according to manufacturer instructions.
- Immerse the Harvard Applier OptiCaps® completely in the cleaning solution and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid
- Clean critical parts of the Harvard Applier OptiCaps® for 1 Minute with a soft brush.
- Move all moveable parts several times in the cleaning solution.
- Leave the Harvard Applier OptiCaps® in the cleaning solution for at least 5 minutes.
- For completely removal of the cleaning solution rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.

Check the Harvard Applier OptiCaps® on cleanliness. If necessary, repeat all above steps.

Manual disinfection

Disinfection with alcohol or 1,5 % glutaraldehyd-containing mixtures for disinfection or sterilization is possible (max. 1 hour).

- Prepare the disinfectant to manufacturer instructions.
- Immerse the Harvard Applier OptiCaps® completely in the disinfectant and carry out all further steps below the liquid level to prevent splashing of the contaminated liquid.
- Move all moveable parts several times in the cleaning solution.
- For exposure time refer to manufacturer instructions.
- For completely removal of the disinfectant rinse the dispenser under running tap water (drinking water quality). Move all moveable parts several times.
- Wipe with a lint free tissue or dry with clean compressed air.

Drying

- Allow the Harvard Applier OptiCaps® to dry until visibly dry.

Maintenance, Inspection and Testing

Inspect the Harvard Applier OptiCaps® under normal lighting:

- Any Harvard Applier OptiCaps® showing visible damage or wear must not longer be used.
- Repeat manual cleaning and inspection if the Harvard Applier OptiCaps® is not visibly clean.

Sterilization

The Harvard Applier OptiCaps® can be sterilized by hot steam (134°C/273 °F for 3 – 3.5 minutes).

- Place the Harvard Applier OptiCaps® in the sterilizer chamber.
- Start the sterilization program.
- At the end of the sterilization program remove the Harvard Applier OptiCaps® and let it cool down.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

Sterilize the capsule applier prior to use.

Warranty

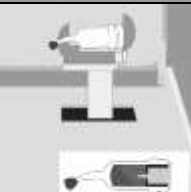
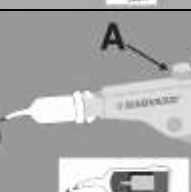
Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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