Harvard Block'n Mask Flow

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Harvard Block'n Mask Flow, 2 x 1 ml Spritze, 8 Needle Tips

Instructions for Use

Flowable composite for blocking and masking

Product description

Harvard Block'n Mask Flow is a flowable light cure radiopaque composite that is used in combination with Harvard MultiChrome or Harvard MultiChrome Flow to mask discolored teeth.

Harvard Block'n Mask Flow contains dimethacrylate based resins and inorganic fillers < 0.2 µm. The total filler content is 67 % by weight and 57 % by volume. Radiopacity of Harvard Block'n Mask Flow is 150 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine: ~100 % aluminium).

Indications/Intended use

- Masking of the oral cavity in case of class III and IV cavities
- Masking of tooth discoloration

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application

1. Preparation of tooth

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation.

2. Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

3. Application of a bonding agent

Apply a common adhesive for enamel and dentine bonding according to the corresponding manufacturer instructions. The application of the universal adhesive Harvard InterLock® ONE or the self etching Harvard InterLock® are recommended. Follow the corresponding instructions.

4. Application and curing of Harvard Block'n Mask Flow

In case of cavities of dark or discolored teeth use Harvard Block'n Mask Flow to ensure that the restoration may not appear dark.

Apply Harvard Block'n Mask Flow by using suitable instruments. Matrix strips may be used.

Preparation of the syringe:

Screw off the cap of the syringe and substitute it by one of the supplied needle tips (by turning clockwise).

Store the screw cap!

After use discard the needle tip and attach the cap to the syringe again. For hygienic reasons needle tips are for single use only.

Apply Harvard Block'n Mask in **layers of max. 1 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400-500 nm) with a **light intensity of at least 1000mW/cm²**, cure each increment separately for **20 seconds**.

Hold the light emission window as close as possible to the filling material.

Finish the restoration with Harvard MultiChrome or Harvard MultiChrome Flow according to the instructions.

5. Additional informations/warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Composition

Dimethacrylates, prepolymerized resin, dental glass, silicon dioxide, photo initiators

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices ([EUDAMED](https://ec.europa.eu/tools/eudamed) – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard Block'n Mask Flow conforms to the DIN EN ISO 4049, type 1, class 2, group 1.

Keep the instructions for use for the duration of the application.

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