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Instructions for Use EN**Light-curing self-etching
one component adhesive****Product description**

Harvard Bond SE Mono is a self-etching light cure one-component adhesive.

The simple to use material is applied on moist tooth surfaces and guarantees a long lasting and safe adhesion between dentin /enamel and the light cure restorative.

Harvard Bond SE Mono is compatible with all visible light cure methacrylate based composite restorative materials.

Indications/Intended use

Adhesive for:

- Restorations with light cure composites

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, Harvard Bond SE Mono may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application**1. Isolation**

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

4. Application of Harvard Bond SE Mono

Apply Harvard Bond SE Mono with a Microbrush® onto the **moist enamel and dentin surfaces for 30 seconds** with agitation.

The material should build a homogeneous layer. Air thin gently for **10 seconds** to remove the volatile components and to disperse the

adhesive. Take care, that the adhesive is not removed from the tooth surface by a strong air stream.

Do not touch the treated surface!

Then light cure with a suitable polymerization unit (wavelength range 400–500 nm, light intensity of at least 500mW/cm²) for **20 seconds** before placement of a light cure composite.

5. Placement of the restorative

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite (e.g. Harvard PremiumFlow) followed by the application of a moldable composite (e.g. Harvard PremiumFill) according to the instructions.

6. Additional informations/Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Composition

Hydrophilic methacrylates, MDP, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.**Article**

Harvard Bond SE Mono,
5 ml bottle
7083601
7095156 50 Microbrush®, refill bag with 50 applicators

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Microbrush® is not a registered trade mark of Harvard Dental International GmbH.