Harvard Bond TE Mono

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Light curing total-etch bonding

Product description

Harvard Bond TE Mono is a simple to use light cure one component bonding agent for the total etch technique with a phosphoric acid etching gel („Etch and Rinse“). It is designed for strong bonding of light cure composites to etched enamel and dentine and to nonprecious and precious metals.

Good, long lasting adhesive strength and good biocompatibility are attained by polycarboxylic acids.

Harvard Bond TE Mono is compatible with all current brands of visible light cure methacrylate-based composite materials.

It is ethanol-based and hydrophilic. Harvard Bond TE Mono can be used on slightly moist dentin surfaces (wet bonding technique).

Indications/Intended use

Adhesive for:

- Restorations with light cure composites

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, Harvard Bond TE Mono may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard Calcium-Hydroxide).

4. Application of Harvard Bond TE Mono

4.1. Enamel and Dentine Conditioning

According to the total etch technique apply a 37% phosphoric acid etching gel (e.g. Harvard Etch) onto the enamel and dentine surfaces beginning with the enamel bevels. Condition the enamel for **at least 15 seconds** and the dentine for **15 seconds or less**. (This results in 20–30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer. Rinse for **20 seconds** with water. Dry it in a water-free and oil-free airstream, but do not desiccate. A slightly wet dentine surface is important for the function of Harvard Bond TE Mono. The etched enamel bevel should have a chalky white appearance.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

4.2. Application of Harvard Bond TE Mono

Apply Harvard Bond TE Mono generously with a brush onto the enamel and dentin surfaces, for **30 seconds** with agitation. The material should build a homogeneous layer. Remove excess material carefully. Dry cautiously with oil free air for about **15 seconds** to remove all volatile components and to disperse the adhesive to an even layer. Do not desiccate the dentine.

Note:

If not used immediately, place dispensed Harvard Bond TE Mono in subdued light to prevent premature polymerization by incident light. Harvard Bond TE Mono will **not** selfcure.

Then light cure with a suitable polymerization unit (wavelength range 400–500 nm, light intensity of at least 500mW/cm²) for **20 seconds** before application of a second layer of Harvard Bond TE Mono.

4.3. Application of a Second Layer of Harvard Bond TE Mono

Apply again the Harvard Bond TE Mono generously with a brush onto the adhesive surfaces as described under **point 3.2**, before placement of a light cure composite.
Do not touch the treated surface!

5. Placement of the restorative

Apply the restorative material according to the instructions of the manufacturer.

Best results are obtained with application of a thin layer of a light cure flowable composite (e.g. Harvard RestoreFlow) followed by the application of a moldable composite (e.g. Harvard Restore) according to the instructions.

6. Additional informations/Warnings

- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Composition

Hydrophilic methacrylates, modified acrylic acids, photo initiators

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, user exclusively remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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