
Harvard CalciumHydroxide LC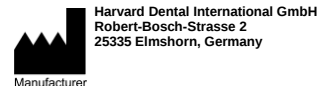

Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de

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Artikel
Harvard CalciumHydroxide LC
7071263 2 ml Spritze, 3 Needle Tips

Instructions for Use EN

Calcium hydroxide paste, light cure

Product description

Harvard CalciumHydroxide LC is a light cure, radiopaque liner with calcium hydroxide.

Indications/Intended use

- indirect pulp capping
- lining of cavities for following filling procedures

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, the material may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor allow not polymerized material to come into contact with materials containing eugenol.

Application

Clean and dry the cavity thoroughly. Avoid contamination.
Rubber dam is the recommended method of isolation.

Note:

In direct proximity to the pulp, separated by a thin layer of dentine only, application of an aqueous calcium hydroxide liner (e.g. Harvard CalciumHydroxide) or Harvard MTA is recommended.

To enhance bonding of Harvard CalciumHydroxide LC use a bonding agent. Etching with Harvard Etch and application of Harvard Bond TE Mono according to the corresponding user instructions is recommended. In areas with a thin dentin layer over the pulp do without bonding (pulp protection).

Then Harvard CalciumHydroxide LC can be applied directly from the syringe. Replace the cap of the syringe by a Harvard NeedleTip H18.

Apply Harvard CalciumHydroxide LC in a thin layer (max. 1 mm) and light cure for **40 seconds** with a suitable polymerization unit (wavelength range 400–500 nm, light intensity of at least 1000mW/cm²). Hold the light emission window as close as possible to the filling material.

Proceed with placement of a light cure composite (e.g. Harvard PremiumFill) according to the corresponding instructions.

After use discard the needle tip and attach the cap to the syringe again.

Needle tips are for single use only due to hygienic reasons!

Additional information/Warnings

- When using Harvard CalciumHydroxide LC wear protective glasses and cover the patient eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- Unpolymerized materials may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin and mucous membrane. If accidental contact occurs, wash immediately with water.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Do not use any resin to adjust viscosity of composite restorative material.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Composition

Dimethacrylates, dental glass, calcium hydroxide, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED- <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental Internationals sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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