Harvard Implant Semi-permanent

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Instructions for Use EN

Dual-curing, Semi-permanent luting cement

Product description

Harvard Implant Semi-permanent is a dual cure Semi-permanent luting cement. Harvard Implant Semi-permanent is based on multifunctional methacrylic esters and contains zinc.

Harvard Implant Semi-permanent is suitable for Semi-permanent cementing of implant abutment retained crowns and bridges. It shows good elastic behavior and good seal thanks to controlled slight expansion. Harvard Implant Semi-permanent is delivered in 4:1 Minmix syringes, so it is very easy and comfortable to use. Harvard Implant Semi-permanent does not influence the setting of resin-based filling and luting materials.

Indications/intended use

- Cementing of crowns and bridges on implants

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, Harvard Implant Semi-permanent may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the material. Neither store the material in proximity of eugenol containing products, nor allow not polymerized material to come into contact with materials containing eugenol.

APPLICATION

Preparing the Minmix-Syringe

Harvard Dental International GmbH only recommends for Harvard Implant Semi-permanent the use of Harvard mixing cannulas Harvard Mini 4/10:1 S-Brown.

Remove the cap of the Minmix-syringe and throw it away (do not use it again!). It is replaced by a 4:1 mixing cannula. Bleed the Minmix-syringe before applying the mixing cannula. Gently press the plunger until both components (base and catalyst) begin to flow out evenly. Make sure that the guidance of the Minmix-syringe is aligned with that of the mixing cannula and turn the cannula 90° clockwise until it locks in position. The material is now ready for application.

Note:

Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Store used syringe with fixed used mixing cannula in a dark place.

Cementing

Before cementing degrease, clean (e.g. with alcohol) and dry abutment and restoration.

Harvard Implant Semi-permanent is automatically mixed when dispensed with slight and even pressure. It can be applied directly into the restoration.

Apply a thin layer of Harvard Implant Semi-permanent to the inside of the restoration. Position the restoration within 2 minutes (working time at 23 °C) cautiously but firmly over the implant abutments.

For a fast fixation of the restoration cure Harvard Implant Semi-permanent at the marginal regions for 20 seconds per surface. In case of translucent materials it is also possible to expose to light through the material (40 seconds per surface). Without any light Harvard Implant Semi-permanent sets in the mouth within approx. **4 minutes**. After Harvard Implant Semi-permanent has set remove any excess.

Notes

If the restoration has to be removed for inspection, this must be done gently with slight and permanent traction.

Avoid jerky removal! The remaining cement can be extracted almost in one piece. Use water spray and suction to remove any remaining residue.

Clean and disinfect abutment and restoration with a cotton pellet soaked in alcohol or another disinfectant. Rinse thoroughly with water and dry.

Now the restoration can be fixed again with Harvard Implant Semi-permanent (see Cementing).

Additional Notes/Warnings

- Contact of resin pastes with skin should be avoided, especially by anyone having known allergies to methacrylates.
- In case of contact with skin and eyes rinse immediately with plenty of water.
- If swallowed seek medical advice.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-20 °C (39-68 °F). Protect from direct sunlight. Do not use after expiry date.

Composition

Dimethacrylates, zinc-containing fillers, silicon dioxide, catalysts, photo initiators

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7081400	5 ml Minmix-Spritze with 10 mixing cannulas 4:1 Shade universal
7093050	Harvard Mini 4/10:1 S-Brown Refill bag with 50 mixing cannulas

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