Harvard InterLock® ONE

Harvard Dental International GmbH
Robert-Bosch-Strasse 2
25335 Elmshorn, Germany

Tel: +49 (0) 30/99 28 978-0
Fax: +49 (0) 30/99 28 978-19
info@harvard-dental-international.de
www.harvard-dental.de

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Light cure universal adhesive

Product description

Harvard InterLock® ONE is a simple to use light cure universal adhesive. Harvard InterLock® ONE can be used with the self-etch technique, with preceding selective enamel etching or with the total-etch technique. It is suitable for moist, wet and dry surfaces.

For applications, where light cure cannot be ensured (e.g. in the root canal) Harvard InterLock® ONE has to be used in combination with Harvard InterLock® ONE Self Cure Activator. Please refer to the instructions for Harvard InterLock® Self Cure Activator.

Indications/Intended use

Adhesive for the self-etch technique, selective enamel etching or total-etch technique for:

- Direct light cure composite and compomer restorations.
- Core build-ups with light, dual and self cure composites (e.g. Harvard ZirconCore) if light cure of the adhesive is possible.
- Cementing of inlays, onlays, crowns and bridges with light, dual and self cure composite cements if light cure of the adhesive is possible.
- Repair of fractured composite restorations

Use Harvard InterLock® ONE only with dual cured or self cured composites if light cure of the adhesive is guaranteed.

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In singular cases, Harvard InterLock® ONE may cause a sensitizing reaction in patients with a hypersensitivity to any of the ingredients. In these cases, the material should not be used. Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application

1. Isolation

Rubber dam is the recommended method of isolation.

2. Cavity Preparation

Prepare the cavity according to the principles of the adhesive restoration technique.

3. Pulp Protection

For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

4. Conditioning with Harvard Etch (optional)

Selective enamel etching or etching by application of the total-etch-technique can be used to improve the bond to enamel.

Note:

Unprepared enamel surfaces must always be conditioned with Harvard Etch.

Etching precaution:

It is essential, that etched areas are not contaminated by anything. If contamination occurs, re-etch, rinse with water and

dry as above. Avoid etching gel contact with oral soft tissues, eyes and skin. If accidental contact occurs, flush immediately with copious amounts of water.

a. Selective enamel etching

Apply Harvard Etch onto the enamel surfaces and leave in place for **at least 15 seconds**. Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water and dry it in a water-free and oil-free airstream. The etched enamel surface should have a chalky white appearance.

b. Total-Etch Technique

Apply Harvard Etch onto the enamel and dentine surfaces beginning with the enamel bevels. Condition **the enamel for at least 15 seconds and the dentine for 15 seconds**. (This results in 20–30 seconds etching of enamel and 15 seconds of dentine). Deciduous teeth are etched correspondingly longer. Rinse for 20 seconds with water. Dry it in a water-free and oil-free airstream, but do not desiccate. The etched enamel surface should have a chalky white appearance.

5 Application of Harvard InterLock® ONE

Bottle:

Dispense one or two drops Harvard InterLock® ONE into a mixing well and use immediately after dispensing. **Close bottle immediately after use.**

SingleDose:

Make sure that the ampoule is opened just before use. Hold the snap-off ampoule at each end. (place thumb below the center score). Press firmly until the ampoule separates. Discard the cap. Take up the adhesive with a suitable brush under rotation and apply.

Apply Harvard InterLock® ONE with the Microbrush® onto the whole enamel and dentin surface in a thin and even layer. Keep the adhesive agitated for **at least 20 seconds**. Applying the adhesive on the tooth surface without agitation is inadequate. The material should build a homogeneous layer. Air thin gently (**at least 5-10 seconds**) to remove the volatile components and to disperse the adhesive until a glossy and uniform layer results. Then light cure with a suitable dental halogen light unit or an LED (wavelength 400–500 nm, light intensity min. 1000 mW/cm²) for **20 seconds**.

6. Application of a composite

Refer to manufacturer instructions for placement of the light, dual or self cure restorative material.

Note for direct restorations:

Best results are obtained with application of a thin layer of a light cure flowable composite followed by the application of a moldable composite. Light cure each composite layer separately according to the corresponding user instructions.

7. Additional informations/Warnings

- Harvard InterLock® ONE SingleDose is for single use only due to hygienic reasons..
- Unpolymerized material may have an irritating effect and may lead to a sensitizing reaction against methacrylates.
- Avoid contact with skin, mucous membrane and eyes.
- If the material comes into contact with skin, immediately wash with water and soap. If the material comes into contact with eyes, immediately rinse with copious amounts of water and seek medical advice if required.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Composition

Hydrophilic methacrylates, MDP, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7083611	Harvard InterLock® ONE
7083613	5 ml bottle
7083612	50 SingleDose ea. 0,1 ml,
7095156	Self Cure Activator 5 ml bottle
	Microbrush®, refill bag with 50 applicators

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