Harvard LiquiCord®

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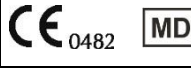
Retraction paste for temporary gingival displacement

Product description

Harvard LiquiCord® is a retraction paste for temporary displacement of the marginal gingiva.

Harvard LiquiCord® is applied directly from the syringe into the sulcus. This results in a dry, clean and opened sulcus.

Depending on the clinical situation and the working technique, Harvard LiquiCord® can be used alone or in combination with retraction cords or other retraction agents.

Indications/Intended use

Harvard LiquiCord® is for all indications for temporary displacement of the marginal gingiva and to provide for a dry sulcus if the periodontium is healthy:

- Taking conventional impressions
- Taking digital impressions
- Cementing of restorations (temporary and permanent)
- Preparing of class II and V fillings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Harvard LiquiCord® must not be used in patients with diseased periodontium, open furcations or exposed bone.

Patient target group

Persons who are treated during a dental procedure.

Intended users

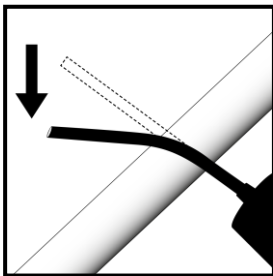
This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation

Immediately before starting with one of the above listed working procedures take the syringe from the bag.

Bend one of the supplied needle tips to the desired angle but avoid to bend sharply (narrow). This can be done preferably with the **round** handle of a dental instrument.

Screw off the screw cap of the syringe and substitute it by the needle tip.

Note: Ensure that the needle tip is firmly in position on the syringe!

Dispense a small quantity of Harvard LiquiCord® onto a mixing pad and discard.

2. Application of Harvard LiquiCord®

Before application of Harvard LiquiCord® rinse the sulcus thoroughly with water and dry slightly with air.

Introduce the tip of the needle tip into the sulcus, opening the sulcus. Move the top of the tip slowly and evenly around the tooth while pressing out the paste. Fill the sulcus all around with enough Harvard LiquiCord® so that an excess appears.

If necessary, Harvard LiquiCord® can be used in combination with a retraction cord, that is placed before in the sulcus, to open the sulcus further.

Optional Harvard LiquiCord® can be used together with the Harvard Sulcus Former. Therefore place the Harvard Sulcus Former onto the teeth and let the patient bite with moderate pressure. This results in better moisture control around the preparation and helps to press Harvard LiquiCord® into the sulcus. Let Harvard LiquiCord® take effect for at least 2 minutes and keep away all moisture from the site for this time.

Note: A disproportional long residence time may cause damage of tissue.

After the residence time remove the Harvard LiquiCord® completely from the sulcus with air and water and with the aid of a suction device. The patient must not swallow the retraction paste.

Finally carefully check the sulcus of the treated teeth and the surrounding areas. Remove any residues of Harvard LiquiCord® from the mouth.

Additional information

- In case of contact with eyes wash immediately with plenty of water and consult a physician if necessary.
- Remove any residual paste from metal parts immediately with water.
- Keep away from children!

Storage

Store at 10-25 °C (50-77 °F). Protect from moisture! Do not use after expiry date.

The Harvard LiquiCord® syringe is for single use only.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Caution:

Federal law restricts the sale of this device to or by the order of a dentist.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7083870	Harvard LiquiCord® 10 syringes ea. 0,7 g, 20 needle tips H18 white
7083875	Harvard Needle Tips H18 white refill bag with 50 pcs.