Harvard MTA - DirectSeal

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Harvard MTA - DirectSeal

Ready-to-use, bioactive MTA root sealer

Product description

Harvard MTA - DirectSeal is a ready-to-use bioactive root canal sealer based on MTA for permanent root canal fillings that can be applied directly into the root canal. Harvard MTA - DirectSeal has a high pH value of 12-13 and is used in combination with gutta-percha.

Harvard MTA - DirectSeal is an insoluble, radiopaque one component material which sets due to the physiological liquid present in the root canal.

Indications/Intended use

- Permanent sealing of the root canal in combination with gutta-percha

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Sealer Do not use Harvard MTA - DirectSeal in patients with a known allergy to any of the components of the product. An allergic reaction may require re-treatment.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

Working time

No mixing is required.

Harvard MTA - DirectSeal can be applied immediately and introduced directly into the root canal. The working time at room temperature is about **1 hour**.

Setting Time:

Setting time is about **4-5 hours**. However, in very dry root canals, the setting time can be more than 10 hours.

The setting time of Harvard MTA - DirectSeal depends on the presence of physiological liquid in the dentin, which is naturally present in the dentin. Therefore, it is not necessary to add moisture in the root canal prior to placing the material.

Root canal sealing

Use of rubber dam is recommended

Prepare and clean the root canal (e.g. with Harvard Glide & Clean).

Dry the canal using paper points.

Screw off the cap of the syringe and attach one the supplied EndoTips to the syringe by turning it clockwise.

After use of the syringe replace disposable EndoTip by the screw cap.

EndoTips are for single use only due to hygienic reasons.

Inject Harvard MTA - DirectSeal passively into the root canal, taking into account the previously determined working length. Filling should occur from bottom upwards by pulling the

syringe tip slowly out of the root canal. Insert a gutta percha into the canal up to the working length.

Using a heating device to cut the remaining excess of the gutta percha at the coronal part. Condense the gutta-percha for a better sealing with a plugger.

Seal the cavity tightly.

Additional remarks

- Store Harvard MTA - DirectSeal in the original aluminium bag.
- Keep away from children!

Composition

MTA, radiopaque mineral oxide

Storage

Store at 10-25 °C (50-77 °F) in the original aluminium bag and protected from moisture. After each use place the syringe back in the aluminium bag and seal it well. Use within 6 months after first opening. Do not use after expiry date.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Caution:

Federal law restricts the sale of this device to or by the order of a dentist.

Keep the instructions for use for the duration of the application.

Order-No.

7081514

Article

Harvard MTA - DirectSeal,
1 g syringe, 8 endo tips