Harvard MTA-Retro OptiCaps®

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Instructions for Use

Bioactive MTA cement for retrograde root end filling, extra radiopacity, extra fast

Product description
Harvard MTA-Retro OptiCaps® is a biocompatible endodontic cement in OptiCaps®. Due to its firm consistency in combination with a short setting time MTA-Retro is the ideal choice for **root-end filling (retrograde)**. Harvard MTA-Retro powder consists of very fine hydrophilic particles of several hydraulic mineral oxides. After contact with the liquid it forms a gel that hardens to an impermeable barrier. Harvard MTA-Retro OptiCaps® are easily activated and the content of the capsule is easily ejected with the Harvard Applier OptiCaps®. Mixing of Harvard MTA-Retro OptiCaps® (mixing time 30 seconds) is achieved by a high frequency mixer with about 4,300 oscillations/minute.

Indications/Intended use

- Root-end filling (retrograde)

Performance features
The performance features of the product meet the requirements of the intended use.

Contraindications
None known

Patient target group
Persons who are treated during a dental procedure.

Intended users
This medical device should only be used by a professionally trained dental practitioner.

Application

1. Activation and mixing
See "Instructions for the activation and mixing of Harvard OptiCaps®"
Mixing time for the Harvard MTA-Retro OptiCaps® is 30 seconds.

Attention:
Avoid lag times between the processes of activation, mixing and application as the material is in the process of setting and lag times may impair or prevent application of the material. The material must be extruded within **10 seconds** after the end of mixing. To prevent dehydration during setting, apply Harvard MTA-Retro immediately after mixing.

2. Working time
Working time at 23° C (from the start of mixing):
about 2 minutes

3. Root-end filling (retrograde)
Create a surgical access to the root-end and resect the root. Prepare an apical cavity to a depth of 3-5 mm. Isolate the area and dry the root-end cavity with paper points. Achieve hemostasis with suitable methods. Mix a capsule of Harvard MTA-Retro OptiCaps® as described under item 1 and extrude it on a glass plate. Apply Harvard MTA-Retro with suitable instruments and condense it using a small plugger. Remove excess cement and clean the surface of the root with a moist piece of gauze. Confirm placement of Harvard MTA-Retro repair material with an X-ray. Harvard MTA-Retro remains as a permanent part of the root canal filling.

Additional Notes/Warnings

- In the first hour after application handle the placed MTA cement carefully.
- Store Harvard MTA-Retro OptiCaps® in the sealed packaging in a dry place prior to use.
- Harvard MTA-Retro can cause discoloration.
- Keep away from children!

Composition
MTA, radiopaque mineral oxide

Storage
Store Harvard MTA-Retro at 2 – 25 °C (36 °F – 77 °F) in the original seal-pack and protected from moisture. Do not use after expiry date. Harvard MTA-Retro OptiCaps® are for single use only.

Disposal
Disposal of the product according to local authority regulations.

Reporting obligation
Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note
The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty
Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability
Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order.-No.	Article
7081506	Harvard MTA-Retro 2 OptiCaps® ea. 0.25 g, ea. packed in an aluminium pouch
7092000	Harvard Applier OptiCaps®

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