Harvard MultiChrome Multi Shade Restorative

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Product description

Harvard MultiChrome is a light cure radiopaque composite for anterior and posterior restorations. Harvard MultiChrome can be used for all natural tooth shades due to its continuous color matching properties. Beside its permanent gloss and toothlike opalescence Harvard MultiChrome shows excellent physical properties and good handling features. High gloss polishability demands only little effort.

Harvard MultiChrome contains dimethacrylate based resins and inorganic fillers < 0.2 µm. The total filler content is 80 % by weight and 75 % by volume. Radiopacity of Harvard MultiChrome is 200 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).

Harvard Block’n Mask is a light cure composite that is used in combination with Harvard MultiChrome to mask discolored teeth. Harvard Block’n Mask contains dimethacrylate based resins and inorganic fillers < 0.2 µm. The total filler content is 80 % by weight and 76 % by volume. Radiopacity of Harvard Block’n Mask is 200 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).

Indications Harvard MultiChrome

- Restorations of all cavity classes
- Direct veneers
- Diastema closure
- Repair of ceramic restorations (e.g. veneers, crowns)

Indications Harvard Block’n Mask

- Masking of the oral cavity in case of extended class III and IV cavities
- Masking of tooth discoloration

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application

1. Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation.

2. Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

3. Application of a bonding agent

Apply a common adhesive for enamel and dentine bonding according to the corresponding manufacturer instructions. The application of the universal adhesive Harvard InterLock® ONE or the self etching Harvard InterLock® are recommended. Follow the corresponding instructions.

4. Application and curing of Harvard MultiChrome and Harvard Block’n Mask

In case of deep and extended cavities best results are obtained with a build-up of a base liner using a light cure resin modified cement (e.g. Harvard Ionoresin Fill Extra) or a light cure flowable composite (e.g. Harvard UltraFill Flow), followed by the application of Harvard MultiChrome. In case of small cavities Harvard MultiChrome can be applied directly.

Note:

In case of extended class III and class IV cavities use Harvard Block’n Mask palatinal to ensure that the restoration may not appear dark. Use Harvard Block’n Mask also as a first layer for class I, II and V cavities of dark or discolored teeth.

In case of repair of ceramic or composite restorations use a light cure opaquer as first layer in case of metal or zirconia surfaces.

Apply Harvard Block’n Mask in **layers of max. 1 mm**. Light cure each increment separately. By using a polymerization unit (wavelength range 400-500 nm) with a **light intensity of at least 1000mW/cm²**, cure each increment separately for **20 seconds**.

Hold the light emission window as close as possible to the filling material.

Apply Harvard MultiChrome in **layers of max. 2 mm**. Light cure each increment separately. Harvard MultiChrome is easy to model. Place it by using a suitable instrument. Matrix strips may be used. By using a polymerization unit (wavelength range 400-500 nm) with a **light intensity of at least 1000mW/cm²**, cure each increment separately for **20 seconds**.

Hold the light emission window as close as possible to the filling material.

5. Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs. Polish to high gloss e.g. with polishing discs or polishing brushes.

6. Additional informations/Warnings

- If composites in OptiTips® are applied directly into the mouth, for hygienic reasons the OptiTip® is for single use only.
- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust the viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized composite may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Composition

Dimethacrylates, prepolymerized resin, dental glass, silicon dioxide, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International GmbHs sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard MultiChrome und Harvard Block’n Mask conform to DIN EN ISO 4049, type 1, class 2, group 1.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7100001	Harvard MultiChrome, 3 g syringe
7100002	Harvard MultiChrome, OptiTips® 20 x 0,25 g
7100005	Harvard Block’n Mask, 3 g syringe
7100007	Harvard Block’n Mask, OptiTips® 20 x 0,25 g
7100009	Harvard MultiChrome Kit 4 x 3 g syringe
7095200	Harvard Applier OptiTips®

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