Harvard PremiumSil Mono MaxiMix

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Instructions for Use EN

Vinylpolysiloxane impression material, super hydrophilic

Product description

Harvard PremiumSil Mono MaxiMix is a high hydrophilic one phase impression material for crowns, bridges, inlays, partial and complete dentures and denture repairs. It can also be used as impression material for relining, functional impressions, impressions for prosthetic restorations and for impressions using the copper band technique.

Harvard PremiumSil Mono MaxiMix is a medium viscosity hydrophilic precision impression material based on addition curing vinyl silicones delivered in automix-cartridges. It can be extruded directly onto the teeth or the tray.

Harvard PremiumSil Mono MaxiMix shows strong hydrophilic properties. Therefore it improves the wettability and yields an exact reproduction of subgingival areas and other moist surfaces.

Harvard PremiumSil Mono MaxiMix is thixotropic. That means it stays in the position in the tray and on the tooth, but under pressure it flows.

Harvard PremiumSil Mono MaxiMix meets the requirements of ISO 4823, type 2 (medium-bodied consistency).

Indications/Intended use

- Monophase impressions for crowns, bridges, inlays, partial and complete dentures
- Relining, functional impressions, impressions for prosthetic restorations and for impressions using the copper band technique

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation of the cartridge

Harvard Dental International GmbH only recommends for Harvard PremiumSil Heavy MaxiMix the use of mixing cannulas type Harvard Maxi 5:1 Dynamic.

Hold the cartridge with the sealed outlet openings facing upwards. Take the handle of the sealing cap and remove the sealing cap upwards. Avoid excessive force. Insert the cartridge into the mixing machine (according to manufacturer instructions). Attach a new mixing tip to the cartridge. After the mixing tip is correctly seated (the central hexagonal socket at the lower end of the tip must be properly aligned with the hexagonal drift shaft), place the yellow bayonet ring over the mixing tip and push it down completely. Turn the ring clockwise (1/4 turn) to lock the mixing tip securely in place. Check whether the hexagonal drift shaft does engage when starting. The spiral in the mixing tip must rotate when starting.

If the used mixing machine offers different mixing speed levels, use always the lowest mixing speed.

Note:

Discard the first 3 cm of impression material extruded from newly filled cartridges. Store used cartridge with fixed used mixing tip. It serves as a cap.

To remove the used mixing tip turn the bayonet ring anti-clockwise and draw it off.

2. Impression taking

Load the auto mixed material onto the tray. Insert the tray not later than **2:00 min** (working time at 23°C or 74°F) after extruding Harvard PremiumSil Mono MaxiMix into the mouth. Maintain position for at least **3:00 minutes**.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones.

Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation

shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.

7083723

380ml maximix cartridge, 10 dynamic mixers, 1 bayonet ring

7091400

Harvard Maxi 5:1 Dynamic Refill bag with 50 dynamic mixers and 2 bayonet rings

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