Harvard RestoreFlow

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Light cure universal flowable microhybrid composite

Product description

Harvard RestoreFlow is a visible light cure easy flowable microhybrid composite for anterior and limited posterior restorations and for sealing. Harvard RestoreFlow is based methacrylate-resin and inorganic filler particles of 0.05-1 µm. The filler content: is 62 % by weight and 47 % by volume. Radiopacity of Harvard RestoreFlow is 150 % aluminium (radiopacity of enamel: ~200 % aluminium, of dentine:~ 100 % aluminium).

Indications/Intended use

- Class V restorations (cervical caries)
- Anterior class III and IV restorations
- All kind of small restorations
- Lining of cavities
- Sealings

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Irritations resulting from direct contact with the pulp cannot be ruled out. Therefore for pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Incompatibility with other materials

Do not use in combination with substances containing eugenol because eugenol inhibits the polymerization of the composite. Neither store the composite material in proximity of eugenol containing products, nor allow not polymerized composite to come into contact with materials containing eugenol.

Application

1. Preparation of tooth and color matching

Clean the tooth with a fluoride-free polishing paste (e.g. flour of pumice with water) prior to preparation and color matching.

Ascertain the tooth shade while teeth are still moist and select the appropriate Harvard RestoreFlow shade.

Harvard RestoreFlow is shaded according to VITA®-Shades.

2. Cavity preparation

After isolation (best with a rubber dam) prepare the cavity with minimal tooth reduction. For pulp protection areas close to the pulp should be covered with a thin layer of pulp capping material (e.g. Harvard BioCal®-CAP or Harvard CalciumHydroxide).

3. Application of a bonding agent

Condition and apply a bonding agent according to the total etch technique (e.g. Harvard Etch) and subsequent application of a light cure bonding agent (e.g. Harvard Bond TE Mono).

Alternatively a self etching bonding agent e.g. Harvard InterLock® (two component adhesive) resp. Harvard Bond SE Mono (one component adhesive) can be applied according to the corresponding instructions.

4. Application and curing of Harvard RestoreFlow

Apply Harvard RestoreFlow in the selected shade. Place it by using suitable instruments. Transparent matrix strips may be used.

Preparation of the syringe:

Screw off the cap of the syringe and substitute it by one of the supplied needle tips.

Store the screw cap!

After use discard the needle tip and attach the cap to the syringe again. For hygienic reasons needle tips are for single use only.

For an optimum result apply Harvard RestoreFlow in **layers of max. 1 mm**. Light cure each increment separately.

By using a polymerization unit (wavelength range 400 – 500 nm) with a light intensity of at least 1000 mW/cm², cure each increment as follows:

Lighter shades (e.g. A1, A2, A3) **20 seconds**

Darker shades (e.g. A3.5) **30 seconds**

Hold the light emission window as close as possible to the filling material.

5. Finishing

Depending on the shape of the area contour with finishing diamonds, flexible grinding discs or carbide burs.

Polish to high gloss e.g. with polishing discs or polishing brushes.

6. Additional informations/Warnings

- The ambient light of the dental lamp may start polymerization of the composite.
- Do not use any resin to adjust viscosity of composite restorative material.
- Avoid contact with skin, mucous membrane and eyes.
- Unpolymerized Harvard RestoreFlow may have an irritant effect and can lead to sensitization against methacrylates.
- Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- VITA® is a registered trade mark of the VITA-Zahnfabrik, Bad Säckingen, Germany.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date. Close syringe immediately after use to avoid exposure to light.

Composition

Dimethacrylates, dental glass, silicon dioxide, photo initiators.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard RestoreFlow conforms to the DIN EN ISO 4049, type 1, class 2, group 1.

Keep the instructions for use for the duration of the application.

Order-No.	Article
Harvard RestoreFlow, 3 ml syringe, 3 needle tips	
7083211	Shade A1
7083212	Shade A2
7083213	Shade A3
7083214	Shade A3.5
7091226	Harvard Needle Tips H20 refill bag with 50 pcs.

Harvard Dental International GmbH
Robert-Bosch-Strasse 2
25335 Elmshorn, Germany

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