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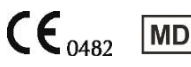
## Harvard RootTemp

### Calcium hydroxide for temporary root canal dressing

#### Product description

**Harvard RootTemp** is a radiopaque water-based calcium hydroxide paste.

#### Indications/Intended use

- temporary filling of root canals (endodontic dressing)
- direct pulp capping when the pulp is opened
- indirect capping for the treatment of caries profunda

#### Performance features

The performance features of the product meet the requirements of the intended use.

#### Contraindications

- Direct pulp capping after exposure of the pulp in dentin if:
  - the area of exposure is larger than 1 mm<sup>2</sup>
  - a completely dry field cannot be established
  - the treatment of the tooth cannot be completed in one appointment
  - the tooth concerned is already causing discomfort
- direct and indirect pulp capping if the pulp is damaged irreversible
- the material should not be used in patient with a known allergy to any of the ingredients

#### Patient target group

Persons who are treated during a dental procedure.

#### Intended users

This medical device should only be used by a professionally trained dental practitioner.

#### Preparation of the syringe

Screw off the cap and substitute it by a flexible endo tip. **Store the screw cap!**

#### Handling precaution

Squeeze out the first small drop on a pad to ensure proper handling and prevent "shooting" out of the Harvard RootTemp by entrapped air. This must be done for each new application.

#### Application

Clean and dry the cavity.

#### Temporary filling of root canals

Prepare and clean the root canal (e.g. hydrogen peroxide 3% or sodium hypochlorite solution) and insert Harvard RootTemp paste with the flexible endo tip directly from the syringe. Avoid overfilling (apical) and trapping of air.

#### Pulp capping

Apply Harvard RootTemp small-area (pinpoint) with the flexible endo tip directly from the syringe onto the opened pulp resp. onto the area close to the pulp. The applied material can be dried carefully with an oil and water free stream of air. Do not apply Harvard RootTemp to the cavity edges and remove any paste sticking to them.

Apply a suitable adhesive according to the manufacturer instructions and proceed with the restoration.

#### Important Notes

- The material hardens when exposed to air.
- Immediately after use replace the cap.
- For each application take **a new flexible endo tip. The flexible endo tips are for single use only due to hygienic reasons!**
- Before placing a new flexible endo tip remove hardened paste from the syringe outlet.

- **Do not use hardened paste.** If hardening still occurs inside the flexible endo tip, use a new one.

#### Side-effects

If the root canal is overfilled, discomfort may result (sensitivity for pressure and/or swelling) but, as a rule, this will subside within 24-48 hours.

#### Warnings

- When using Harvard RootTemp wear protective glasses and cover the patient eyes.
- Harvard RootTemp is irritant. In case of contact with eyes a risk of serious damage to eyes exists.
- Avoid contact with skin, mucous membrane and eyes.
- In case of contact with eyes, rinse with copious amounts of water and consult a physician immediately.
- If accidental skin contact occurs, wash immediately with water.
- Keep away from children!

#### Storage

Store at 8 – 25 °C (46 – 77 °F). Do not store at less than +8° C (46 °F)!

Do not use after expiry date.

#### Composition

Calcium hydroxide.

#### Disposal

Disposal of the product according to local authority regulations.

#### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

#### Note

The summary of safety and clinical performance of the medical device can be found in the European database on medical devices (EUDAMED – <https://ec.europa.eu/tools/eudamed>).

#### Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

#### Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

#### Order-No.

7071264

7083618

#### Article

Harvard RootTemp

2 ml syringe, 6 flexible endo tips

Refill bag with 25 endo tips

 Manufacturer	Harvard Dental International GmbH Robert-Bosch-Strasse 2 25335 Elmshorn, Germany	 Made in Germany	Tel: +49 (0) 30/99 28 978-0 Fax: +49 (0) 30/99 28 978-19 info@harvard-dental-international.de www.harvard-dental.de	V02_MDR 2025-09