Harvard SuperSnap® Putty Soft MaxiMix Regular Snap

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Vinylpolysiloxan-Abformmasse, Putty Soft-Konsistenz

Produktbeschreibung

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Instructions for Use EN

Vinylpolysiloxane impression material, Putty Soft consistency

Product description

Harvard SuperSnap® Putty Soft MaxiMix Regular Snap is a base material.

It is characterized by a long working time in conjunction with a short intraoral setting time (super snapset).

Harvard SuperSnap® Putty Soft MaxiMix is a precision impression material based on addition curing vinyl silicones delivered in MaxiMix-cartridges. It can be used with suitable mixing machines.

Harvard SuperSnap® Putty Soft MaxiMix shows optimized detail accuracy.

Harvard SuperSnap® Putty Soft MaxiMix meets the requirements of ISO 4823, , type 0 (putty- consistency).

Indications/Intended use

- Double-mix technique (1 step/2 phase) with Harvard SuperSnap® Light Regular Snap or Harvard SuperSnap® Medium Regular Snap Correction impression technique (2 step/2 phase) with Harvard SuperSnap® Light or Harvard SuperSnap® Medium

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparation of the cartridge

Harvard Dental International GmbH only recommends for Harvard SuperSnap® Putty Soft MaxiMix the use of mixing cannulas type Harvard Maxi 5:1 Dynamic.

Hold the cartridge with the sealed outlet openings facing upwards. Take the handle of the sealing cap and remove the sealing cap upwards. Avoid excessive force. Insert the cartridge into the mixing machine (according to manufacturer instructions). Attach a new mixing tip to the cartridge. After the mixing tip is correctly seated (the central hexagonal socket at the lower end of the tip must be properly aligned with the hexagonal drift shaft), place the yellow bayonet ring over the mixing tip and push it down completely. Turn the ring clockwise (1/4 turn) to lock the mixing tip securely in place. Check whether the hexagonal drift shaft does engage when starting. The spiral in the mixing tip must rotate when starting.

If the used mixing machine offers different mixing speed levels, use always the lowest mixing speed.

Note:

Discard the first 3 cm of impression material extruded from newly filled cartridges.

Store used cartridge with fixed used mixing tip. It serves as a cap.

To remove the used mixing tip turn the bayonet ring anti-clockwise and draw it off.

2. Impression taking

Double-mix technique

Load the impression tray with Harvard SuperSnap® Putty Soft MaxiMix Regular Snap and apply Harvard SuperSnap® Light Regular Snap or Harvard SuperSnap® Medium Regular Snap around the preparation. Tray loading and syringing around the preparation have to be completed at the same time. Insert the tray not later than **1:45 minutes** from start of mixing (working time at 23°C or 74°F) into the mouth. Maintain position for at least **1:45 minutes**.

Two-step (correction) impression technique

Load the impression tray with Harvard SuperSnap® Putty Soft MaxiMix Regular Snap. Insert the loaded tray in the mouth not later than **1:45 minutes** (working time at 23°C or 74°F) after start of mix. Maintain position for at least **1:45 minutes**. Cut drain channels into the pre-impression.

Load the auto mixed Harvard SuperSnap® Light or Harvard SuperSnap® Medium onto the tray with the preliminary impression and/or directly around the teeth. Position the tray within the working time of the correction impression material into the mouth. Maintain position until hardening has finished.

Refer to the corresponding user instructions.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones. Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not use with standard (condensation reaction type) silicone impression materials.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- When using the double-mix technique (1 step/2 phase), make sure that both materials have the same setting time in mouth. The working time is determined by the material with the shorter working time.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7083853	Harvard SuperSnap® Putty Soft MaxiMix, Regular Snap, black 380ml maxmix cartridge, 10 dynamic mixers, 1 bayonet ring
7091400	Harvard Maxi 5:1 Dynamic Refill bag with 50 dynamic mixers and 2 bayonet rings

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