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Instructions for Use

EN

Vinylpolysiloxane impression material, Putty Soft consistency

Product description

Harvard SuperSnap® Putty Soft Regular Snap is a base material.

It is characterized by a long working time in conjunction with a short intraoral setting time (super snapset).

Harvard SuperSnap® Putty Soft is a precision impression material based on addition curing vinyl silicones.

Harvard SuperSnap® Putty Soft shows optimized detail accuracy.

Harvard SuperSnap® Putty Soft meets the requirements of ISO 4823, type 0 (putty-consistency).

Indications/Intended use

- Double-mix technique (1 step/2 phase) with Harvard SuperSnap® Light Regular Snap or Harvard SuperSnap® Medium Regular Snap
- Correction impression technique (2 step/2 phase) with Harvard SuperSnap® Light or Harvard SuperSnap® Medium

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

None known

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Mixing

Perforated or non-perforated metal or plastic trays may be used with Harvard SuperSnap® Putty. In order to improve adhesion of Harvard SuperSnap® Putty to the tray, a silicone adhesive (e.g. Harvard TraySive) should be applied according to the corresponding instructions.

Use equal parts (volume or weight) of base paste and catalyst paste. Take the base paste with the dark measuring spoon and the catalyst paste with the white measuring spoon. Knead until the color is homogeneous (at least **30 seconds**).

2. Impression taking

Double-mix technique

Load the impression tray with Harvard SuperSnap® Putty Soft Regular Snap and apply Harvard SuperSnap® Light Regular Snap or Harvard SuperSnap® Medium Regular Snap around the preparation. Tray loading and syringing around the preparation have to be completed at the same time.

Insert the tray not later than **1:45 minutes** from start of mixing (working time at 23°C or 74°F) into the mouth. Maintain position for at least **1:45 minutes**.

Two-step (correction) impression technique

Load the impression tray with Harvard SuperSnap® Putty Soft Regular Snap. Insert the loaded tray in the mouth not later than **1:45 minutes** (working time at 23°C or 74°F) after start of mix. Maintain position for at least **1:45 minutes**. Cut drain channels into the pre-impression.

Load the auto mixed Harvard SuperSnap® Light or Harvard SuperSnap® Medium onto the tray with the preliminary impression and/or directly around the teeth. Position the tray within the working time of the correction impression material into the mouth. Maintain position until hardening has finished.

Refer to the corresponding user instructions.

3. Casting

The impression should be stored under normal room conditions (max. 25°C or 77°F). The impression can be poured 30 minutes after removal from the mouth. No other time limitation does exist.

4. Electroplating

All impressions can be copper- or silverplated.

5. Disinfection

Impressions can be disinfected with disinfection solutions specific for impression materials based on addition curing silicones.

Refer to the manufacturer instructions.

Storage

Do not store above 25 °C (77 °F). Protect from direct sunlight. Do not use after expiry date.

Additional notes:

- Do not interchange lids of pots. Do not interchange measuring spoons.
- Do not use Latex or sulfur-containing polymeric gloves, they may interfere with the setting reaction of the impression material.
- When using the double-mix technique (1 step/2 phase), make sure that both materials have the same setting time in mouth. The working time is determined by the material with the shorter working time.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's

application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

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