Harvard TEMP Cem EasyDose®

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7081103 10 ml EasyDose®-Spritze

Instructions for Use EN

Eugenol-free temporary cement for cementing crowns and bridges

Product description

Harvard TEMP Cem EasyDose® is a non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. Harvard TEMP Cem EasyDose® shows excellent retention to temporary crown and bridge materials, but can be easily removed from prepared teeth.

EasyDose® shows excellent adhesion to temporary crown and bridge materials, but can be easily removed from prepared teeth. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on zinc oxide and organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Indications/intended use

- Temporary cementing of provisional crowns or bridges
- Trial cementing of permanent crowns and bridges

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use Harvard TEMP Cem EasyDose® in cases where known allergies to the components exist.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparing the EasyDose®-syringe

Remove the cap of the EasyDose®-syringe and put it aside. (Do not throw it away!)

When using the syringe the first time, press the plunger gently until the material flows evenly from both orifices and discard this material.

Extrude the desired amount of Harvard TEMP Cem EasyDose® with slight and even pressure onto a mixing pad and mix with a suitable spatula for **30 seconds** until the material gets to a homogeneous consistency.

After use close the EasyDose®-syringe with the cap.

2. Application

Apply a thin layer to the inside of the temporary crown or bridge and place the temporary restoration within **1:30 minutes** (working time at 23 °C/74 °F) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

Harvard TEMP Cem EasyDose® sets in the mouth within **about 2-3 minutes**. After Harvard TEMP Cem EasyDose® has set remove any excess.

Removing of the restoration

Remove the restoration with a suitable instrument. Residual Harvard TEMP Cem EasyDose® has to be removed completely with a suitable instrument (e.g. scaler, probe or hand instrument) before final cementing. Clean dentine and enamel with a moist cotton pellet. Wash thoroughly with water spray and blot dry.

Storage information

Store at 8 – 25 °C (47 - 77 °F). Protect from moisture! Do not use after expiry date.

Additional informations/Warnings

- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard TEMP Cem EasyDose® meets the requirements of ISO 3107 type 1 and the ADA/ANSI Specification No. 30. Keep the instructions for use for the duration of the application.

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