Harvard TEMP Glaze LC

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Harvard TEMP Glaze LC

Light cure one component varnish for temporary crowns and bridges

Product description

Harvard TEMP Glaze LC is a light cure one component acrylic varnish which cures hard without any smear layer. Chemically it is based on multifunctional acrylates.

Indications/Intended use

Surface coating of

- temporary crowns and bridges
- full or partial dentures
- light cure custom trays
- orthodontic devices

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use with patients with known allergy to methacrylates. In case of an allergic reactions please contact a physician.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by professional trained dental/dental technician users.

Properties

The special properties of Harvard TEMP Glaze LC are its full dry surface without leaving a smear layer after light polymerisation, its hard high gloss surface and its more abrasion resistant surface.

The resistance of the covered device will even be improved against solvation by some denture cleaning agents and against deposition of food rests and plaque, which is advantageous for patient.

Application

- After creating the acrylic crown wash off the smear layer with a suitable solvent e.g. alcohol.
- Apply a thin layer of Harvard TEMP Glaze LC without inclusion of air bubbles with a brush or with a foam onto the dull surface.
- Place into a light oven suitable for curing light cure composite inlays, onlays, veneers etc, or light cure trays.
- The curing time depends on efficiency of the light cure unit: Stroboscopic light Dentacolor XS (Kulzer) will take about **1½ minutes**, weaker lights up to **3 minutes**.
- Alternatively Harvard TEMP Glaze LC can be cured for **20 seconds** from each side with a Dental-Halogen Light Cure Unit (min. 500 mW/cm²) or a LED Curing Light (min. 1000 mW/cm²).
- After light curing slight or no polishing is indicated. The surface shines high gloss.

Note

Close bottle immediately after use.

Do not apply or cure Harvard TEMP Glaze LC in the mouth!

Additional Notes/Warnings

- Harvard TEMP Glaze LC contains methylmethacrylate.
- Harvard TEMP Glaze LC is highly flammable. Avoid sources of ignition and do not store in direct sunlight.
- Irritates the eyes, respiratory system and the skin.
- Can possibly cause skin sensitization on contact.
- In case of contact with oral tissue or skin, flush immediately with water.
- In case of contact with eyes, flush immediately with water and seek medical attention.
- Wear protective gloves. Commercial medical gloves do not protect against the sensitizing effect of methacrylates.
- Keep away from children!

Storage information

Storage temperature 4-25°C (39-77°F). Protect from direct sunlight. Do not use after expiry date.

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusively remedy and Harvard Dental International's sole obligation shall be repair or replacement of the Harvard Dental International product.

Limitation of Liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Keep the instructions for use for the duration of the application.

Order-No.	Article
7081730	Harvard TEMP Glaze LC 30 ml bottle