Harvard TEMP Cem Esthetic

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Instructions for Use EN

Translucent non-eugenol luting cement for temporary cementation

Product description

Harvard TEMP Cem Esthetic is a translucent non-eugenol temporary crown and bridge cement suitable for cementing temporary crowns or bridges as well as for trial cementing of permanent restorations. It's translucency makes it the perfect choice for esthetic dentistry. Harvard TEMP Cem Esthetic shows excellent adhesion to temporary crown and bridge materials, but can be easily removed from prepared teeth. Harvard TEMP Cem Esthetic is delivered in Minimax syringes, so it is very easy and comfortable to use. Due to its non-eugenol formulation it does not influence the setting of resin-based filling and luting materials. The temporary crown and bridge cement is based on zinc oxide and organic acids. Especially in cases of allergies to eugenol it is strongly recommended to use this eugenol-free type of cement.

Indications/Intended use

- Temporary cementing of provisional crowns or bridges
- Trial cementing of permanent crowns and bridges

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

Do not use Harvard TEMP Cem Esthetic in cases where known allergies to the components exist.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by a professionally trained dental practitioner.

Application

1. Preparing the minimix syringe

Harvard Dental International GmbH only recommends for Harvard TEMP Cem Esthetic the use of mixing cannulas type Harvard Mini 1:1 S-Brown.

Remove the cap of the minimix syringe and throw it away (**do not use it again!**).

Attach a 1:1 mixing tip. Make sure that the guidance of the minimix syringe is aligned with that of the mixing tip and turn the tip 90° clockwise until it locks in position. The material is now ready for application.

Note:

Discard the first 2-3 mm of the extruded material. This must be done for each new mix.

Precautions

Harvard TEMP Cem Esthetic should extrude easily from the Minimax-syringe. Do not use excessive force. Excessive pressure may result in an uncontrolled extrusion of the material and may cause rupture of the syringe.

Leave the used mixing tip on the minimix syringe. It serves as a cap.

2. Application

Apply Harvard TEMP Cem Esthetic directly into the temporary crown or bridge under slight pressure. Apply a thin layer to the inside of the temporary crown or bridge and place the temporary restoration within **1:30 minutes** (working time at 23 °C/74 °F) cautiously but firmly over the prepared tooth structures.

Note:

Do not dry stumps, best moisten them with some water. A humid milieu (humid stumps) additionally facilitates to remove the temporary crown or bridge later on.

Harvard TEMP Cem Esthetic sets in the mouth within **about 2-3 minutes**. After Harvard TEMP Cem Esthetic has set remove any excess.

Removing of the restoration

Remove the restoration with a suitable instrument. Residual Harvard TEMP Cem Esthetic has to be removed completely with a suitable instrument (e.g. scaler, probe or hand instrument) before final cementing.

Clean dentine and enamel with a moist cotton pellet. Wash thoroughly with water spray and blot dry.

Storage information

Store in the original seal-pack at 8-25 °C (47-77 °F). After opening of the seal-pack, use Harvard TEMP Cem Esthetic within 3 months and before the end of expiry date. Avoid constantly high humidity. Do **not** store in the refrigerator. Do not use after expiry date.

Additional informations/Warnings

- In case of contact with skin and eyes rinse immediately with plenty of water and consult a physician if necessary.
- If swallowed seek medical advice.
- Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

Harvard Dental International GmbH warrants this product will be free from defects in material and manufacture. Harvard Dental International GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user's application. If this product is defective within the warranty period, your exclusive remedy and Harvard Dental International GmbH's sole obligation shall be repair or replacement of the Harvard Dental International GmbH product.

Limitation of liability

Except where prohibited by law, Harvard Dental International GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

Harvard TEMP Cem Esthetic meets the requirements of ISO 3107 type 1 and the ADA/ANSI Specification No. 30. Keep the instructions for use for the duration of the application.

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